Installation manual

MANUEL D'INSTALLATION
INSTALLATIEHANDLEIDING
INSTALLATIONSANLEITUNG
MANUALE DI INSTALLAZIONE
MANUAL DE INSTALACIÓN
INSTRUKCJA INSTALACJI
PŘÍRUČKU PRO INSTALACI

Moost

CE

Preface	4
1. Your product	5
1.1. Compatibility	6
2. Scope of delivery	6
3. Assembly	7
3.1. Foldable wheelchairs	7
3.2. Trigo wheelchairs	8
3.3. Fixed-frame wheelchairs (option 1)	12
3.4. Fixed-frame wheelchairs (option 2)	14
3.5. Assembling the Moost drive module and operating console	16
3.6. Disassembly	16

Preface

To support you on the installation of the Moost device, this manual is offered. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact Vermeiren.


Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

On our website <http://www.vermeiren.com/> you will always find the most recent version of the following information. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual of the wheelchair For user and specialist dealer
	User manual of Moost For user and specialist dealer
	Installation manual of Moost For specialist dealer
	EC declaration of conformity

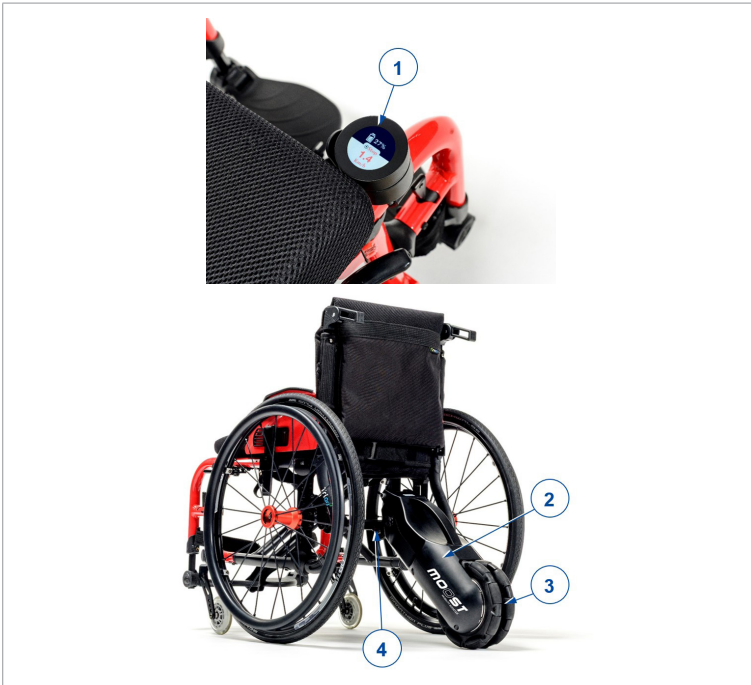1. Your product

1. Operating console
2. Moost device
3. Driving wheels
4. Mounting mechanism

1.1. Compatibility

The information in this manual applies to the installation of Moost on all wheelchairs in the following list:

- Sagitta
- Sagitta SI
- Trigo S
- Trigo T
- V500 Active
- V500 Light
- V500 Light 30°
- TNS wheelchairs:
 - Activator
 - Activator 2.5
 - Proval
 - Activator F.C.
 - RSX

2. Scope of delivery

The following items are part of the delivery.

- Moost drive module
- Operating console
- Tools
- Manual

3. Assembly

⚠ CAUTION

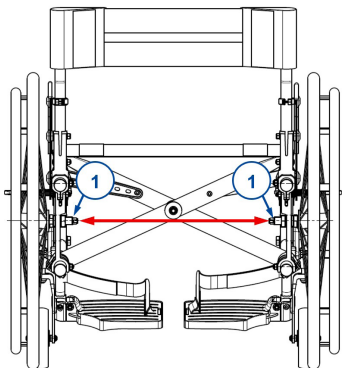
Risk of damage

- Prevent cables from being crushed or bent with kinks. Two hook-and-loop straps are at your disposal to route the cable.


To attach Moost to your wheelchair, the system requires a mounting mechanism depending on the type of wheelchair (foldable wheelchairs, or fixed-frame wheelchairs). The mounting system needs to be configured only once according to your wheelchair's size and settings, after which the Moost system itself can be attached or removed without the need for repeated adjustments.

3.1. Foldable wheelchairs


- i** Applicable to:
- V500 Active
 - V500 Light
 - V500 Light 30°

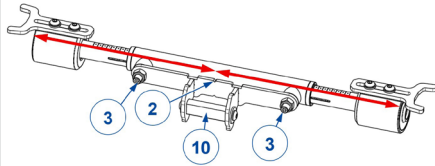1. Measure the distance between the axles of the rear wheels (1).

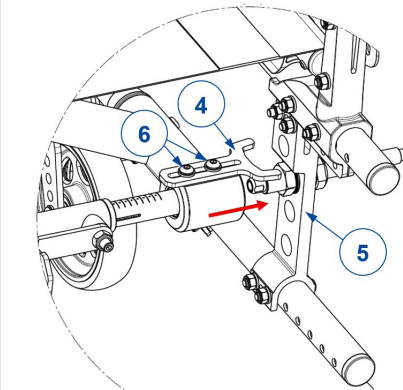
2. Adjust the length of the mounting bar to the measured distance + 2cm. Make sure that the Moost attachment bracket (10) is located in the middle of the bar (use the indication (2) for reference).
3. First tighten the bolts, and then the nuts (3).


4. Place the mounting bar on the axles by pressing the spring on either side of the mounting bar.
5. Place the hooks (4) over the axle plate (5).
6. Tighten the screws (6) once the hooks are correctly placed.

3.2. Trigo wheelchairs

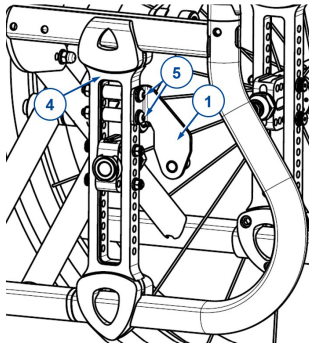
- i** Applicable to:
- Trigo S
 - Trigo T

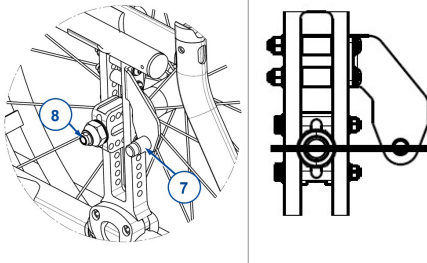
1. Install the brackets (1) on the rear wheel support (4) by tightening the two bolt-bushing-nut connections (5). Make sure that the axle substitutions (7) are on the inside, and on the same level as the actual wheel axles (8).

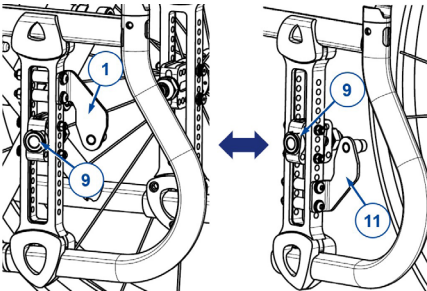
- i** Depending on the position of the rear wheel support (4), the bolt-nut connections (5) may be inverted to allow for easy assembly (by inserting the bolts on the side of the bracket, instead of inserting the bolts on the opposite side).

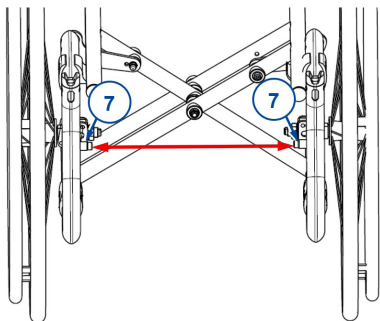
- i** Based on the position of the rear wheel axle block (9), the brackets (1) can be placed above or below the axle block. Regular assembly position is above the axle block (9). If the brackets need to be placed below the axle block, swap the brackets around to opposite sides as compared to regular assembly, and place them upside down (11). See table 1 for more detailed information.

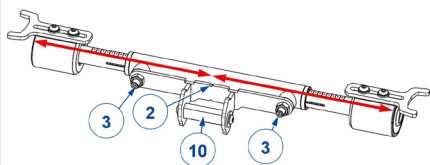
2. Measure the distance between the substitute axles (7).

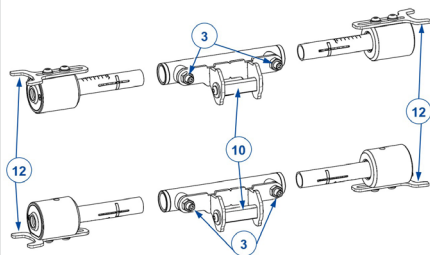3. Adjust the length of the mounting bar to the measured distance + 2cm. Make sure that the Moost attachment bracket (10) is located in the middle of the bar (use the indication (2) for reference).
4. First tighten the bolts, and then the nuts (3).

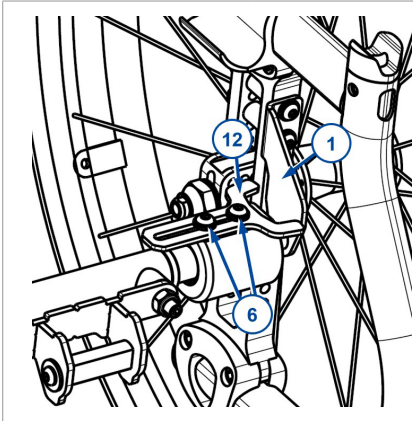- i** If the brackets (1) have been placed upside down in step 1., the hooks (12) need to be placed in upside-down position as well while the attachment bracket (10) remains upright. To do this, remove the screws (3), swap the outer parts around and turn them with hooks upside down. Replace the screws.

5. Place the mounting bar on the axles by pressing the spring on either side of the mounting bar.
6. Place the hooks (12) around the bracket (1).
7. Tighten the screws (6) once the hooks are correctly placed.

Table 1: Trigo wheel position configuration


		Rear wheel support position (front-rear)							
		8	7	6	5	4	3	2	1
Axle block upper bolt position (up-down)	1	B	B	B	B	B	C	C	C
	2	B	B	B	B	B	B	C	C
	3	B	B	B	B	B	B	B	C
	4	B	B	B	B	B	B	B	B
	5	B	B	B	B	B	B	B	B
	6	B	B	B	B	B	B	B	B
	7	B	B	B	B	B	B	B	B
	8	A	A	A	A	A	A	A	C
	9	A	A	A	A	A	A	A	A
	10	A	A	A	A	A	A	A	A
	11	A	A	A	A	A	A	A	A
	12	A	A	A	A	A	A	A	A

A = Bracket above axle block
 B = Bracket below axle block
 C = Excluded


3.3. Fixed-frame wheelchairs (option 1)

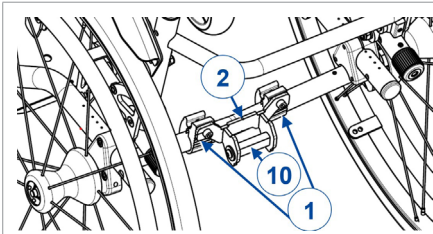
- i** Applicable to:
- Sagitta
 - Sagitta SI

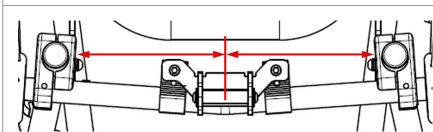
1. Place the clamps (1) on the axle tube. Make sure they can still slide and rotate.

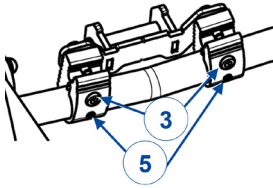
2. Position the attachment bracket (10) in the middle between the wheels (use the indication (2) for reference).

Front

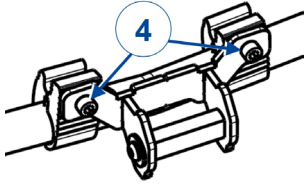Rear

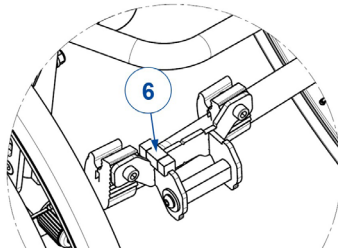
3. Tighten all screws in order: (3) - (4) - (5).


4. Use the spirit level in the provided indentations (6) to make sure that the bracket is level.
5. If necessary, rotate the clamps (1) to the correct, level position. Tighten the screws (3 - 4 - 5).

3.4. Fixed-frame wheelchairs (option 2)


- i** Applicable to:
- Activator
 - Activator 2.5
 - Proval
 - Activator F.C.
 - RSX

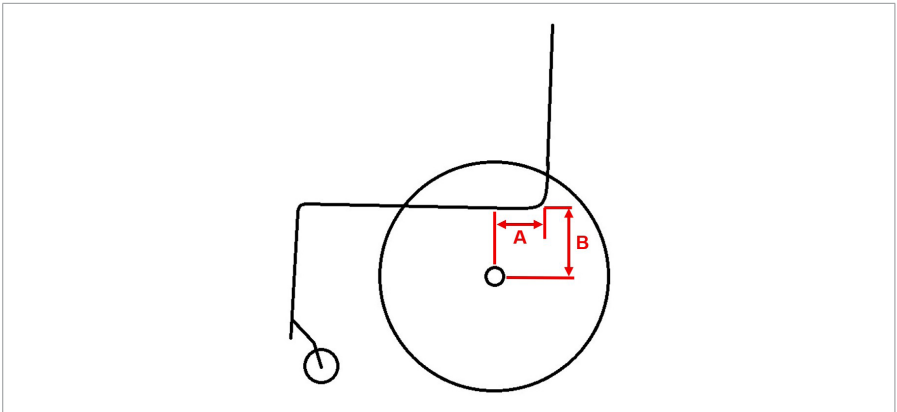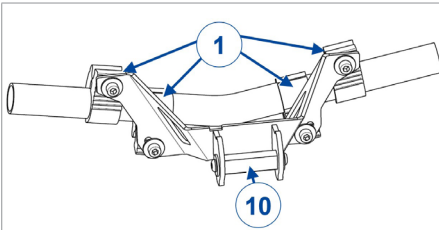- i** This mounting bracket model is only compatible with wheelchairs meeting the following criteria:
- Tube diameter (between rear wheels) = 25,4mm
 - Distance A = max. 80 mm
 - OR if A is more than 80 mm: Distance B = min. 130 mm
 - Rear wheels = 24"
 - Camber = max. 6°
 - Tube between rear wheels: longer than 230 mm

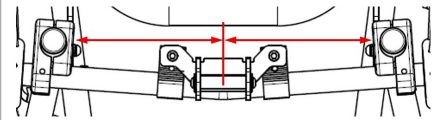
1. Place the clamps (1) on the axle tube. Make sure they can still slide and rotate.

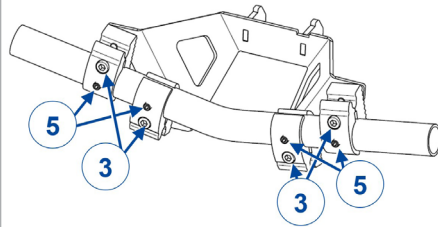
2. Position the attachment bracket (10) in the middle between the wheels.


Front

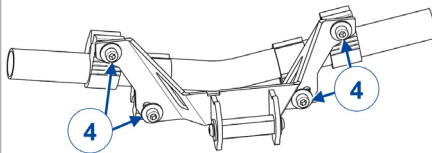3. Tighten all screws in order: (3) - (4) - (5).
Start with the outer clamps, and then the inner clamps.


Rear

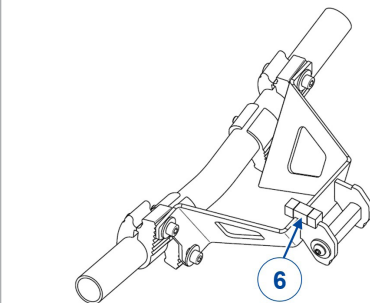4. Use the spirit level in the provided indentations (6) to make sure that the bracket is level.
5. If necessary, rotate the clamps (1) to the correct, level position. Tighten the screws (3 - 4 - 5).

3.5. Assembling the Moost drive module and operating console

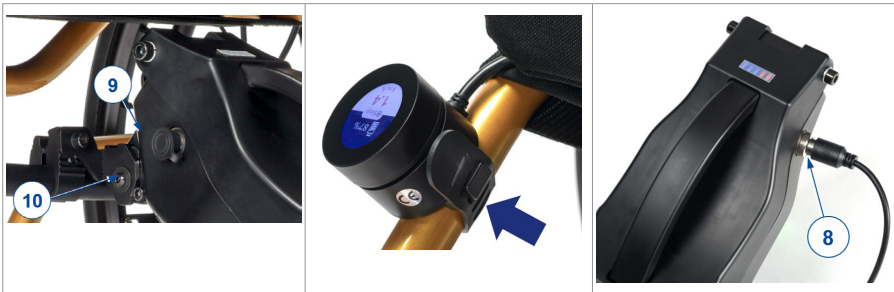1. Tilt the wheelchair back slightly.
2. Slide the Moost receiver (9) over the bracket (10) of the mounting mechanism. Make sure that the bracket is secured inside the receiver when the wheelchair is returned to a normal position (the Moost device is able to turn slightly around the bracket).
3. Using the plastic strap, attach the display to the seat frame in a location comfortable for you.
4. Connect the display cable to the Moost device (8). Screw the securement ring on tight to secure the cable into place.

3.6. Disassembly

1. Switch off the drive electronics.
2. Remove the operating console from the wheelchair frame
or
detach the operating console cable from the Moost drive module (Keep in mind that the cable needs to be kept away from the wheels during use!).
3. Tilt the wheelchair back and remove the Moost device from the bracket (10) of the mounting mechanism.
4. For foldable wheelchairs: If required, remove the mounting system by pressing the spring on either side of the mounting bar.

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All rights reserved, including translation.