Maintenance manual

MANUEL D'ENTRETIEN
ONDERHOUDSHANDLEIDING
WARTUNGSANLEITUNG
MANUALE DI MANUTENZIONE
MANUAL DE MANTENIMIENTO
INSTRUKCJĘ KONSERWACJI
NÁVOD K ÚDRŽBĚ

Beds

Lits

Bedden

Betten

Letti

Camas

Łóżka

Lůžka

Contents

Preface	4
1. Troubleshooting	5
2. Inspection	7
2.1. Test report for medical electrical equipment according to IEC 62353	8
3. Cleaning	11
3.1. Paint and coating	11
4. Disinfection	11

Preface

This maintenance manual is applicable to all Vermeiren beds.

The instructions in this maintenance manual are intended for qualified service personnel.

Vermeiren beds are designed to have an average lifespan of 5 years. Depending on the frequency of use and maintenance, the lifespan of the bed will increase or decrease.

The instructions in this manual are based on and compliant with the requirements of IEC 60601-1, IEC 60601-1-11, IEC 80601-2-52, and IEC 62353.

Servicing and maintenance may only be performed when the device is not in use.

Always unplug the power cable before starting any service, maintenance or repair actions.

The checklists in this manual are used to complete the required maintenance and servicing according to IEC 62353.

Record each session of maintenance, service, repair or cleaning/ disinfection in the *Service Registration Form* at the back of the user manual.

1. Troubleshooting

Symptom	Possible cause	Solution
Bed is not stable or moves	Ground is not flat.	Adjust position.
	Castors are not blocked by brakes.	Apply all 4 brakes, see user manual.
	Brakes are applied but bed can be moved.	Contact your distributor for replacement.
Bed section does not move	End of stroke reached.	Use the handset to move in opposite direction.
	Movement is blocked by lock key.	Unlock the button with the lock key, see user manual.
None of the bed sections move	Bed is not powered.	Plug the power cable in the socket, see user manual.
	Movements are blocked by lock key.	Unlock the buttons with the lock key, see user manual.
	Handset does not work.	Check if handset cable is pinched/damaged. Contact your distributor for replacement.
		Plug the handset in the control box, see installation manual.
Bed section does not stop moving	Handset problem.	Check if button is stuck or damaged. Contact your distributor for replacement.
Wrong bed section moves	Switched motor connections.	Check the cable connections and adjust if necessary, see installation manual.
Control box (CB) power indicator is off	Not connected to the power supply.	Check the connection of the power cable.
	Fuse is damaged.	Contact your distributor for replacement.
	Control box is damaged.	Contact your distributor for replacement.

CB power indicator is off, motor is not working, CB relay makes a clicking sound or makes no sound	Motor cable plug is not well connected to control box.	Connect the motor plug to the control box, see installation manual.
	Motor is damaged.	Contact your distributor for replacement.
	Control box is damaged.	Contact your distributor for replacement.
	Handset is damaged.	Contact your distributor for replacement.
CB power indicator is lit, motor is working, but spindle is not working	Worm gear is loose or worn.	Contact your distributor for replacement.
	Supply voltage is too low.	Supply voltage must be 220V AC.
	Bed is overloaded.	Provide less weight on the bed, and let the motors rest.
CB power indicator is lit, motor and spindle are not working	Cables are not well connected.	Connect the cables.
	Cables are broken.	Replace cable.
	Control box is damaged.	Contact your distributor for replacement.
	Handset is damaged.	Contact your distributor for replacement.
	Motor is damaged.	Contact your distributor for replacement.
CB is working, motor works only in one direction	Handset is damaged.	Contact your distributor for replacement.

i If electrical components are deformed, damaged or not longer operating, contact your distributor for replacement parts.

2. Inspection

The visual inspection and functional testing in this chapter need to be carried out when the device is put into service, after each repair, and after each new installation. It is also recommended to carry out these inspections once every year.

It is advised to carry out the electrical measurements in this chapter once every 3 years.

The frequency may vary depending on the intensity of use.

2.1. Test report for medical electrical equipment according to IEC 62353

Type	Serial number
	Internal ID
Responsible organisation	Name of testing person
<input type="checkbox"/> Test before putting into service, or after new installation <input type="checkbox"/> Recurrent test <input type="checkbox"/> Test after repair	

Visual inspection

Part	Requirement	Pass (P) / Fail (F)
General		
Labelling (identification plate, safety and operating stickers)	Present, clearly readable	
Documentation	Manual available	
Paint and coating	No scratches, no uncoated patches	
Cleanliness	No contamination	
Electrical		
Control unit housing	No damage, no contamination	
Motor housings and lifting tubes	No damage, no contamination	
Handset housing and keypad	No damage, no contamination, correct position	
Cables (handset, motor, power supply,...)	No damage, no contamination, correct position	
Mechanical		
Bed frame	No damage or deformation	
Slatted frame	No damage or deformation	
Castors	No damage or play	
Mattress base	No damage or deformation	
Side rails	No damage, well secured	
Welded seams	No visible cracks	
Wooden parts	No damage, no splinters	

Part	Requirement	Pass (P) / Fail (F)
Screws, bolts, nuts	Securely tightened	
Wear components	No damage or excessive wear	
Pivot points (high-low mechanism)	Well lubricated, no squeaking	
Lifting pole	No damage or deformation	
Comments		

Lubrication

Vermeiren recommends to use PTFE lubrication spray in case of squeaking pivot points. The castor wheels are lubricated for life and do not need additional lubrication.

Functional tests

Part	Requirement	Pass (P) / Fail (F)
Electrical		
Motors	Working smoothly, no abnormal noises; final switch-off	
Handset	Working according to user manual (buttons, locking function)	
Cables (handset, motor, power supply,...)	Well positioned	
Mechanical		
Pivot points, hinges	Working smoothly	
Castors	Brakes working well	
Side rails	Locking well, release according to user manual	
Rastomat	Locking well, release according to user manual	
Manual backrest lowering	Release according to user manual	
Accessories	Securely attached, undamaged	
Comments		

Electrical measurements

The measurements in this chapter need to be performed according to the instructions in IEC 62353 §5.3.

Test	Requirement	Result	Pass (P) / Fail (F)
Leakage currents	According to direct or differential method, for class II Applied Part B or BF: Max. value 100 μ A		

3. Cleaning

CAUTION

Risk of damage by moisture

- Keep handset, motors and connection points clean and protect them from moisture.
- Never use a hose or high-pressure cleaner to clean the bed.

Wipe all rigid parts of the bed with a damp cloth (not drenched). If necessary, use a mild soap, suitable for varnishes and synthetics.

Do not use strong cleaning liquids like solvents, nor use hard brushes.

3.1. Paint and coating

The high quality of the surface layer guarantees optimal protection against corrosion. If the outer coat is damaged by scratching or in some other way, get your specialist dealer to repair the affected surface.

4. Disinfection

CAUTION

Risk of damage

- Disinfections may only be undertaken by authorised persons. Take note of the product information of the products used and wear suitable protective clothing.

All parts of the bed can be treated with a gentle disinfectant and non-abrasive cloth or brush.

The website of the Robert Koch Institute (www.rki.de) contains a list of disinfectants and their recommended concentrations.

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All rights reserved, including translation.