User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

Moost

CE

Preface	4
1. Your product	5
1.1. Compatibility	6
2. Before use	6
2.1. Intended use	7
2.2. General safety instructions	7
2.3. Symbols on the product	8
2.4. First use and storage	9
3. Using Moost	9
3.1. Operating Moost	10
3.2. Battery status and charging	12
3.3. Assembly/Disassembly	14
4. Maintenance	15
4.1. Points of maintenance	15
4.2. Maintenance instructions	16
4.3. Expected lifespan	17
4.4. Reuse	17
4.5. End of use	18
4.6. Warranty	18
5. Troubleshooting	18
6. Technical specifications	19

Preface

Congratulations! You are now owner of a Vermeiren Moost device!

This device is made by qualified and committed personnel. It is designed and produced according to high quality standards, guarded by Vermeiren.

Please read this manual carefully; it will help you to get familiar with the operation, capabilities and limitations of your device.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety, and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.


Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the following information. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual of the wheelchair For user and specialist dealer
	User manual of Moost For user and specialist dealer
	Installation manual of Moost For specialist dealer
	EC declaration of conformity

1. Your product

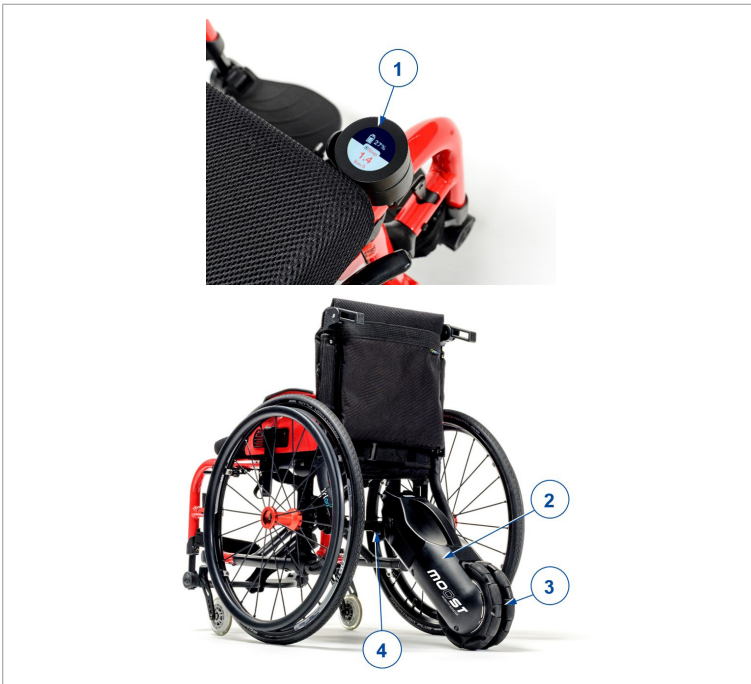1. Operating console
2. Moost device
3. Driving wheels
4. Mounting mechanism

1.1. Compatibility

Moost is compatible with the following wheelchairs:

- Sagitta
- Sagitta SI
- Trigo S
- Trigo T
- V500 Active
- V500 Light
- V500 Light 30°
- TNS wheelchairs:
 - Activator
 - Activator 2.5
 - Proval
 - Activator F.C.
 - RSX

2. Before use

CAUTION

Risk of injuries and damage

- Do NOT install, maintain or use this product unless you have read and fully understood all instructions in this manual and all relevant manuals regarding the wheelchair equipped with this product.
- Do not try to open or disassemble the Moost drive module or operating console in any case - there are no user-serviceable parts inside.
- In case of doubts or questions, do not hesitate to contact your local specialist dealer, your care provider or technical adviser to help you with this.

2.1. Intended use

In this paragraph a brief description of the intended use of your product is given. Additionally, relevant warnings are added to the instructions in the other paragraphs. In this way we would like to make you aware of the possible misuse that may appear.

- This product is an accessory.
- Indications and contraindications: The Moost device is intended to be a propulsion aid for wheelchair users who do not have enough strength to propel the wheelchair themselves during all their activities, or who have to propel the wheelchair for long stretches of time. You should NOT use this device if you suffer from physical or mental impairments that may put you, or other people, in danger when driving the wheelchair.
- Moost is meant to be operated by the wheelchair occupant himself.
- Moost is suitable for indoor and outdoor use.
- Please read all technical details and limits of your Moost device in chapter 6..
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

2.2. General safety instructions

CAUTION

Risk of injuries and/or damage

- Carefully read and follow the instructions in this manual. Otherwise you may get injured or your wheelchair may get damaged.

Keep the following general warnings in mind during use:

- Do not use your wheelchair if you are under influence of alcohol, medicines or other substances that may influence your driving abilities.
- Sources of electromagnetic fields may influence the driving performance of your wheelchair, such as the fields of mobile phones, power generators or high-power energy sources. On the other hand,

the electronics of your Moost system can affect other electronic appliances too, such as alarm systems of shops and automatic doors. We therefore advise you to check your Moost system regularly for damage and wear, since these may enlarge the interference (see also chapter 4.).

Any serious incident [MDR (EU) 2017/745 §2 (65)] that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

2.3. Symbols on the product

	Type designation
	Catalogue number
	Serial number
	Medical device
	Maximum weight of the user in kg
	Manufacturer
	Date of manufacture
	Declaration of conformity
	Attention: important information
	It is advised to read the manual

Maximum speed

EN

2.4. First use and storage

CAUTION

Risk of battery damage

- Never completely discharge the battery.
- Do not interrupt the charging cycle; only disconnect the battery charger when the battery is fully charged.

Ensure that the battery has been fully charged before using your wheelchair, ask your dealer if this has already been done or not. To charge, follow the charging instructions in §3.2..

If the packaging of your product has been damaged, (unintentionally) opened, or affected by environmental conditions (moisture, heat,...) upon delivery, check the device integrity of your product. When in doubt, contact your specialist dealer.

3. Using Moost

WARNING

Risk of injuries

- First read the previous chapters and the user manual of your wheelchair. Do NOT use your wheelchair with Moost unless you have read and fully understood all instructions.
- In case of doubts or questions, do not hesitate to contact your local specialist dealer, your care provider or technical adviser to help you with this.

3.1. Operating Moost

- i** Make sure that you are familiar with the operation of your wheelchair before using it in crowded and possibly dangerous places. First exercise at a spacious, open area with little bystanders.

⚠ WARNING

Risk of injuries or damage


- Do NOT drive on slopes, obstacles, steps or kerbs larger than specified in the wheelchair manual.
- Be aware that your device switches off when the maximum speed is exceeded, for example when descending slopes.
- Pay attention if the road has holes or gaps that may cause entrapment of the wheels.

⚠ CAUTION

Risk of injuries and damage

- Always unplug the battery charger from the wheelchair before starting to drive.
- When driving indoors, only use the lowest speed setting.
- Make sure that your Moost system is switched OFF before getting into/out of the wheelchair.
- Keep fingers and clothes away from the wheel spokes while driving.

3.1.1. Operating console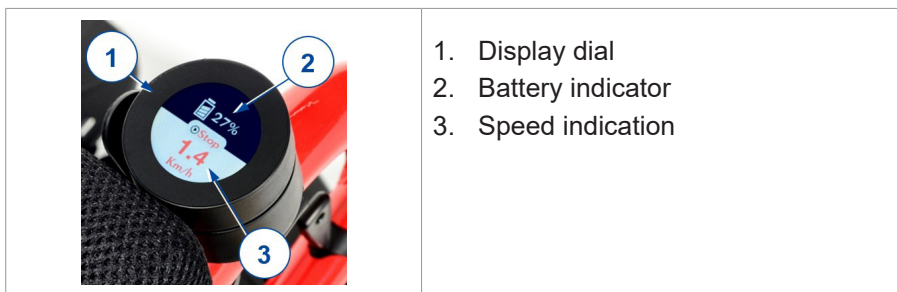

3.1.2. Driving

CAUTION

Risk of injuries

- Moost does NOT brake when propulsion is switched off! Always make sure to manually brake the wheelchair.
1. Make sure that
 - the wheelchair is on a flat surface;
 - the battery is fully charged, see § 3.2.;
 2. Press and hold the display (1) for 2 seconds to switch on the Moost system.
 3. Choose the desired driving speed by turning the display clockwise (faster) or counterclockwise (slower).
-  Turn the display slowly to choose your driving speed. Turning too fast will cause the system to not register your actions well.
4. Shortly press the display to start propulsion. You will start driving at your chosen speed.
 5. If required, turn the display to adjust your speed while driving.
 6. Press the display again to switch off propulsion, or grasp and hold the push rims until propulsion stops.
(If you are driving on slippery surfaces and propulsion doesn't stop immediately when holding the push rims, it helps to move the push rims a bit backward to force a stop.)
 7. Press and hold the display (1) for 2 seconds to switch off the Moost system.

Moost is provided with roll-back resistance, so you cannot manually propel your wheelchair backwards while Moost is switched on. Roll-back resistance also remains active for 100 seconds after switching off the system.

If propulsion is switched off and on again without switching off the Moost system, propulsion will start again at the same speed setting. Speed settings are returned to lowest speed when the system is fully switched off.

It is also possible to start propulsion by manually propelling the wheelchair. In this case, you need to reach 1,5 km/h for electrical propulsion to start, and the speed setting is automatically returned to lowest speed.

3.2. Battery status and charging

WARNING

Risk of injuries and damage due to fire

- Only use the battery charger delivered with the wheelchair. The use of any other charger may be dangerous (fire hazard).
- The battery charger is only intended to charge the batteries delivered with the wheelchair, not to charge any other batteries.
- Do not adapt any of the delivered parts, like cables, plugs or battery charger. Never open or change the battery or the connection points.
- Protect the battery and battery charger against flames, high and low temperatures (see chapter 6.), moisture, sunshine, severe shocks (for example dropping). Do NOT use the battery if this has been the case.
- Do not charge or use a damaged battery, contact your specialist dealer.
- Charge the battery with the battery charger, indoors, in a well-ventilated area, out of children's reach.

CAUTION

Risk of damage

- Battery self-discharge and the quiescent current of the connected users will drain the battery charge slowly. The battery may be irreversibly damaged if the battery is fully discharged. Therefore, make sure that the battery is charged on time.
- Read the storage and maintenance instructions in §4. and technical details in §6..
- Do not charge batteries below 0°C. Move the battery to a warmer place and start charging.
- Keep the connection point for the battery charger free of dust and other contamination.
- In case of any problems due to which the battery cannot be charged according to the following instructions, please contact your specialist dealer.


Instructions to charge the battery:

1. Switch off the Moost system.
2. Connect the charger plug to the charging socket (1).
3. Connect the charger mains plug into the wall socket.
4. The LED indicator on the battery charger will show the status of the charging process:
 - Red - Charging
 - Green - Standby / Fully Charged
5. Once the battery is fully charged, remove the battery charger from the wall socket and the battery.

3.2.1. Battery charge indicator

There are two battery charge indicators: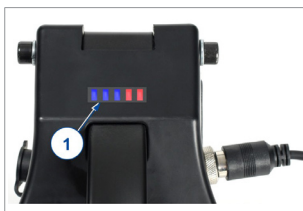

On the Moost device

1. Switch on the Moost system, see §3.1.2..
2. The LED's of the battery charge indicator (1) will indicate the battery charge level.

On the operating console

1. Switch on the Moost system, see §3.1.2..
2. The display shows the battery charge level in percentage (2).

3.3. Assembly/Disassembly

⚠ CAUTION

Risk of damage


- Prevent cables from being crushed or bent with kinks. Two hook-and-loop straps are at your disposal to route the cable.

To attach Moost to your wheelchair, the system requires a mounting mechanism depending on the type of wheelchair (foldable wheelchairs, or fixed-frame wheelchairs). The mounting system needs to be configured only once according to your wheelchair's size and settings, after which the Moost system itself can be attached or removed without the need for repeated adjustments.

For the installation instructions of the mounting mechanism, please consult the installation manual.

3.3.1. Assembling the Moost drive module and operating console

1. Tilt the wheelchair back slightly.
2. Slide the Moost receiver (9) over the bracket (10) of the mounting mechanism. Make sure that the bracket is secured inside the receiver when the wheelchair is returned to a normal position (the Moost device is able to turn slightly around the bracket).
3. Using the plastic strap, attach the display to the seat frame in a location comfortable for you.
4. Connect the display cable to the Moost device (8). Screw the securement ring on tight to secure the cable into place.

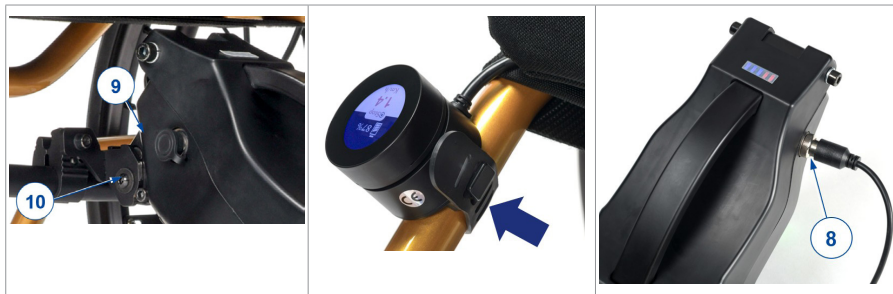3.3.2. Disassembly

1. Switch off the drive electronics.
2. Remove the operating console from the wheelchair frame
or
detach the operating console cable from the Moost drive module (Keep in mind that the cable needs to be kept away from the wheels during use!).
3. Tilt the wheelchair back and remove the Moost device from the bracket (10) of the mounting mechanism.
4. For foldable wheelchairs: If required, remove the mounting system by pressing the spring on either side of the mounting bar.

4. Maintenance

4.1. Points of maintenance

⚠ CAUTION

Risk of injuries and damage

- Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.

- i** The last page of this manual contains a registration form for the specialist dealer to record each service.
The service frequency depends on the frequency and intensity of use. Contact your dealer to agree to a common timetable for inspection/maintenance/repair.

- i** Read the user instructions of the applied battery charger regarding maintenance.

4.1.1. Before each use

Inspect the following points:

- All parts: Present and undamaged or unworn.
- All parts: Clean, see § 4.2.2..
- Operating console, battery charger and relevant cables: no damage such as wires that have become frayed, broken or exposed.
- Battery status: charge the battery when necessary, see §3.2..

Contact your specialist dealer for possible repairs or part replacements.

4.1.2. Yearly or more often

Have your device inspected and serviced by your specialist dealer, at least once a year, or more often. The minimum maintenance frequency depends on use and should therefore be commonly agreed upon with your specialist dealer.

4.1.3. If stored

If your device is stored for a longer period, you still need to charge the battery every month.

4.2. Maintenance instructions

4.2.1. Wheels and tyres

Keep your wheels free of wires, hair, sand and fibres.

4.2.2. Cleaning

CAUTION

Risk of damage by moisture

- Never use a hose or high-pressure cleaner to clean the device.
- Keep the operating console clean and protect it from water and rain.

Wipe all rigid parts with a damp cloth (not drenched). If necessary, use a mild soap, suitable for varnishes and synthetics.

Keep the ventilation openings of the battery charger clean and free of accumulated dust. Blow away the dust and clean the battery charger enclosure with a slightly moistened cloth, if required.

4.2.3. Disinfection

CAUTION

Risk of damage

- Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

4.3. Expected lifespan

The device is designed to have an average lifespan of 5 years. Depending on the frequency of use, driving circumstances and maintenance, the lifespan will increase or decrease.

4.4. Reuse

Before each reuse, have the device disinfected, inspected and serviced according to the instructions in § 4.1. and § 4.2..

4.5. End of use

At end of life, you need to dispose of your device according to the local environmental legislation.

4.6. Warranty

The warranty on this product is subject to the general terms and conditions of each country.

5. Troubleshooting

Even if you use your Moost device properly, a technical problem may occur. In this case, contact your local specialist dealer.

CAUTION

Risk of injuries and damage

- NEVER attempt to repair the Moost device yourself.

This paragraph lists a number of problems with possible solutions. If the problem persists, or if you encounter any other problems not listed here, contact your specialist dealer.

1. The wheelchair does not move when the display is pressed to start propulsion.
 - Verify that Moost is switched on.
 - Verify that the battery is fully charged.
 - Verify that the connection between the display and Moost device is correct.
 - Verify that the wheelchair brakes are not applied.
2. Moost is too fast/slow
 - Try varying the speed settings, see **§3.1.2.**
 - Verify that the wheelchair brakes are not applied.

6. Technical specifications

Brand	Vermeiren
Product group	Electric propulsion unit
Type	Moost
Description	Dimensions
Maximum load (user weight + wheelchair weight) for optimal propulsion assistance in all conditions	100 kg
Total mass (including operating console)	7,3 kg
Energy consumption *	20 km
Dimensions of Moost device (LxWxH)	490 x 150 x 200 mm
Maximum speed forward	6 km/h
Battery	24V 6,6Ah
Driving motors	250W
Noise level	< 65 dB(A)
Degree of protection	IP54
Storage and use temperature	+5°C - +41°C
Operating temperature for electronics	-10°C - +40°C
Storage and use humidity	30% - 70%
We reserve the right to introduce technical changes. Measurement tolerance +- 15 mm / 1,5 kg / 1,5°	
* The theoretical driving distance will be reduced if the wheelchair is used frequently on slopes, rough ground or to climb curbs.	

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Service registration form

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

VERMEIREN GROUP

Vermeirenplein 1/15

2920 Kalmthout

Belgium

www.vermeiren.com

Version: B, 2026-05

Basic UDI-DI: 5415174MoostF5

All rights reserved, including translation.

VERMEIREN
we care for you