Installation manual

MANUEL D'INSTALLATION
INSTALLATIEHANDLEIDING
INSTALLATIONSANLEITUNG
MANUALE DI INSTALLAZIONE
MANUAL DE INSTALACIÓN
INSTRUKCJA INSTALACJI
PŘÍRUČKU PRO INSTALACI

V-series

Contents

Preface	4
1. Scope of delivery	5
2. Tools	6
3. Adjustments	6
3.1. Stability and manoeuvrability	7
3.2. Rear-wheel position	8
3.3. Seat height and angle	9
3.4. Seat height	16
3.5. Seat depth	18
3.6. Parking brakes	25
3.7. Footrest length	27
3.8. Footplate position	29
3.9. Footplate angle	29
3.10. Arm pad height and depth (option 1)	30
3.11. Arm pad depth (option 2)	31
3.12. Armrest height (option 3)	32
3.13. Side plate adjustment (option 4)	33

Preface

To support you on the installation and repairs of this wheelchair, this manual is offered. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact Vermeiren.

The information in this manual applies to the following wheelchair(s):

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Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

On our website <http://www.vermeiren.com/> you will always find the most recent version of the following information. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	Installation instructions (Instructions for detail adjustments) For specialist dealer
	Torque table (maximum torques for bolts and nuts) For specialist dealer
	Service manual for wheelchairs For specialist dealer
	Drawings of (spare) parts For specialist dealer

1. Scope of delivery

The following items are part of the delivery.

- Frame including wheels, backrest frame, seat (default seat height 500 mm, default seat angle 5°)
- 2 Armrests (not for V300XR and V500 Active)
- Seat cushion (only for V300 DL)
- Anti-tipping (only for V300 DL)
- 1 pair of footrests/1 complete footrest
- Tools
- Manual
- Options

Please note that the basic configuration may differ in different European countries. Contact your specialist dealer.

2. Tools

To repair and adjust the wheelchair, the following tools are needed:

- Wrench set no. 7 to no. 22
- Allen keyset no. 3 to no. 8
- Screwdriver Phillips head
- Screwdriver no. 4 to no. 6

3. Adjustments

CAUTION

Risk of injuries or damage

- The wheelchair needs to be adjusted by a specialist dealer according to the instructions in this manual.
 - Any modification to the product will lapse the warranty and will cease the responsibility of Vermeiren.
 - Only use parts and tools described in this manual.
 - Prevent bystanders from accessing the working area.
 - While adjusting and operating the wheelchair, make sure no objects or body parts get caught between moving parts.
 - Only use the settings described in this manual.
 - Make sure that the adjustment on left and right side of the wheelchair are the same.
 - Deviation of the adjustment settings may influence the stability of the wheelchair with risk of tilting and falling.
 - Make sure all screws are firmly secured before riding the wheelchair. Adhere to the tightening torques for screws according to the Torque table on our website.
-  Read the user manual of this wheelchair. Be aware of the technical details and limits of intended use. For more information, contact Vermeiren.

3.1. Stability and manoeuvrability

 Applicable to: V300 and V500 models

CAUTION

Risk of tipping over

- Be aware that by changing the position of the rear wheels, the stability of the wheelchair is influenced. By adjusting the rear wheels
 - in rear adjustment position, the wheelchair becomes more stable, but less manoeuvrable;
 - in front adjustment position, it becomes less stable but more manoeuvrable.

To increase stability or manoeuvrability, the following adjustments can be made:

- Rear-wheel position, see §3.2.
- Seat height and angle, see §3.3.


3.2. Rear-wheel position

i Applicable to: V300 and V500 models

⚠ CAUTION


Risk of injuries or damage

- After adjusting the rear wheels, check the working of the brakes and adjust according to §3.6.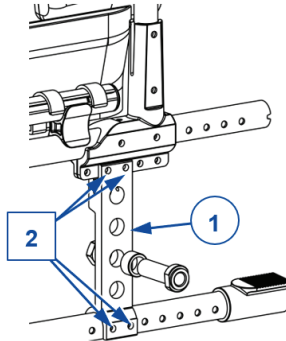

Adjust the position of the rear wheel as follows:

1. Remove the 4 socket screws located in the holes (2) of the mounting plate (1).
2. Move the mounting plate to the front or rear within the range of the adjustment holes.
3. Retighten the 4 socket screws in holes (2).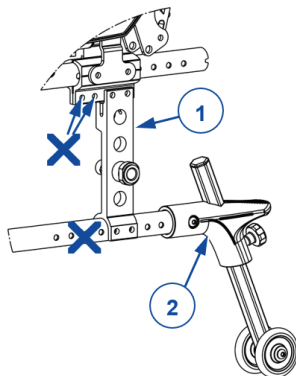

- i** For V300 30°, V500 30°, V500 Light 30°, and depending on use: We advise you to mount the mounting plate (1) of rear wheels in the most rearward position. Install two anti-tipping rollers (2) if the wheelchair is to be moved with inclined backrest.

3.3. Seat height and angle

i Applicable to: V300 and V500 models

3.3.1. Adjustment settings

The seat of the V300 & V500 wheelchairs is adjustable in height and angle (0°-2,5°-5°-7,5°-10°). The adjustment settings depend on the wheelchair model as shown in table 1 up to table 3 in this section. Each table is followed by an illustration of the adjustment points. The adjustment procedures are explained in §3.3.2.

The values in the tables are for the default seat angle of 5°. For other seat angle - seat height combinations, the same table can be used. If you don't find the right seat angle - seat height combination, please contact Vermeiren.



The adjustment of seat height and seat angle is based on:

- Relocating bushing (B) to a position above or under the housing (C) of the castor axle;
- Mounting front wheel higher or lower in the front fork (A);
- Mounting the rear wheel higher or lower in the mounting plate (D).

i Wheelchairs with Bi5 armrests:




Do not install the rear wheels in the highest position (hole 1) to avoid damage to the wheels and armrests.

Table 1: V300DC, V300XL, V300F, V300 DL

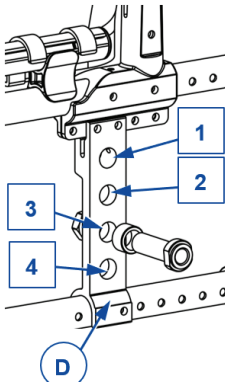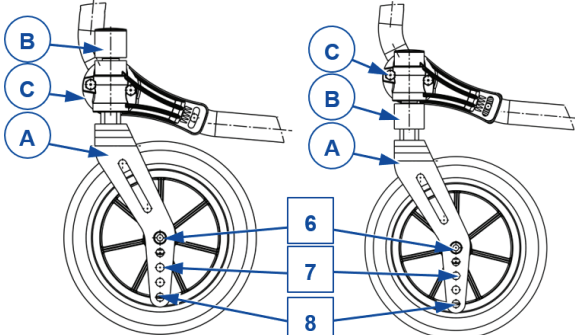Rear wheels	Front wheels	
	Seat height 440 mm	Other seat height
		


At seat angle 5° (Default setting)			
Seat height	Rear wheels	Front wheels	
	Hole	Hole	Bushing (B)
440 mm	1	6	Above (C)
470 mm	2	6	Under (C)
500 mm (Default)	3	7	Under (C)
530 mm	4	8	Under (C)

Table 2: V300XR, V300ACT

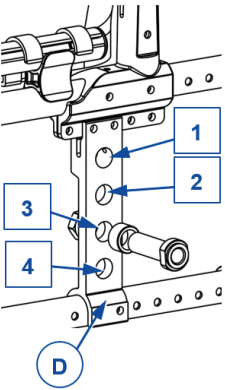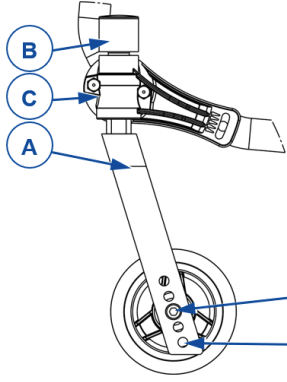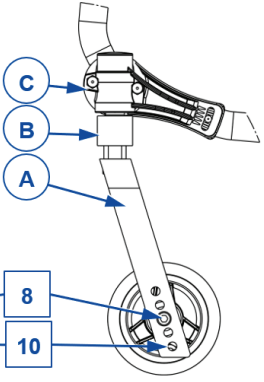Rear wheels	Front wheels	
	Seat height 440 mm	Other seat height
		

At seat angle 5° (Default setting)			
Seat height	Rear wheels	Front wheels	
	Hole	Hole	Bushing (B)
440 mm	1	8	Above (C)
470 mm	2	8	Under (C)
500 mm (Default)	3	10	Under (C)
530 mm (6" wheels)	4	10	Under (C)


Table 3: V300, V500, V500XL, V500 Light, V500 Active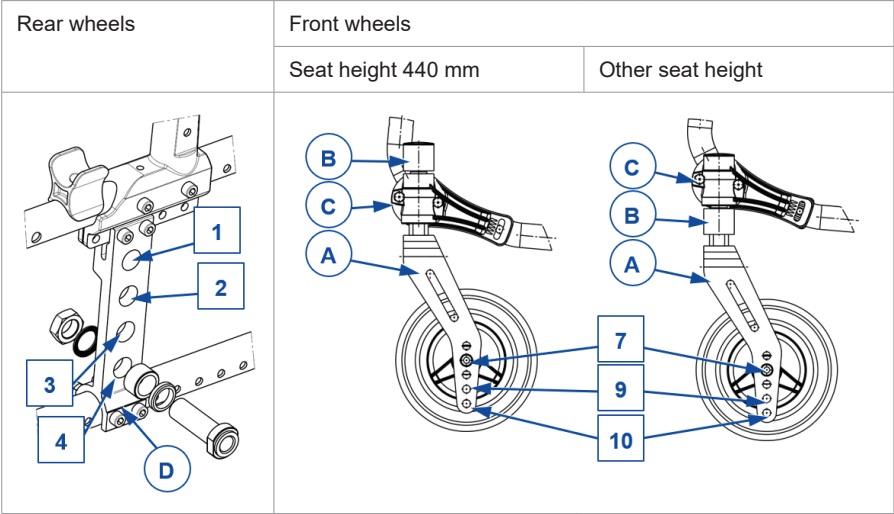

At seat angle 5° (Default setting)				
Seat height	Rear wheels	Front wheels		
	Hole	Hole	Ø	Bushing (B)
440 mm*	1	7	150 mm	Above (C)
470 mm	2	7	150 mm	Under (C)
500 mm (Default)	3	9	150 mm	Under (C)
530 mm (6" wheels)	4	10	200 mm	Under (C)


*V500 Light, V500 Active: 440 mm seat height with 22" rear wheels

3.3.2. Adjusting rear and front wheel height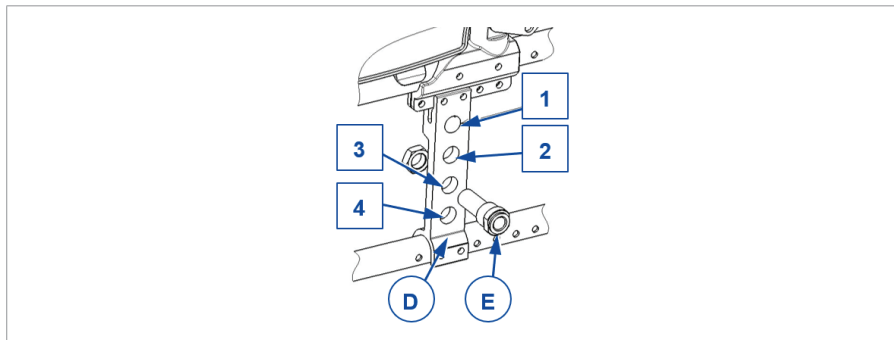

Adjust the rear wheel height as follows:

1. Remove the rear wheels (see instruction manual).
2. Demount the axle housings (E) of the rear wheels from the axle plate (D) and remount in the correct hole, see tables in §3.3.1.
3. Repeat this for the second rear wheel. Make sure both housings are positioned identically.
4. Verify that both axle housings (E) are tightened firmly.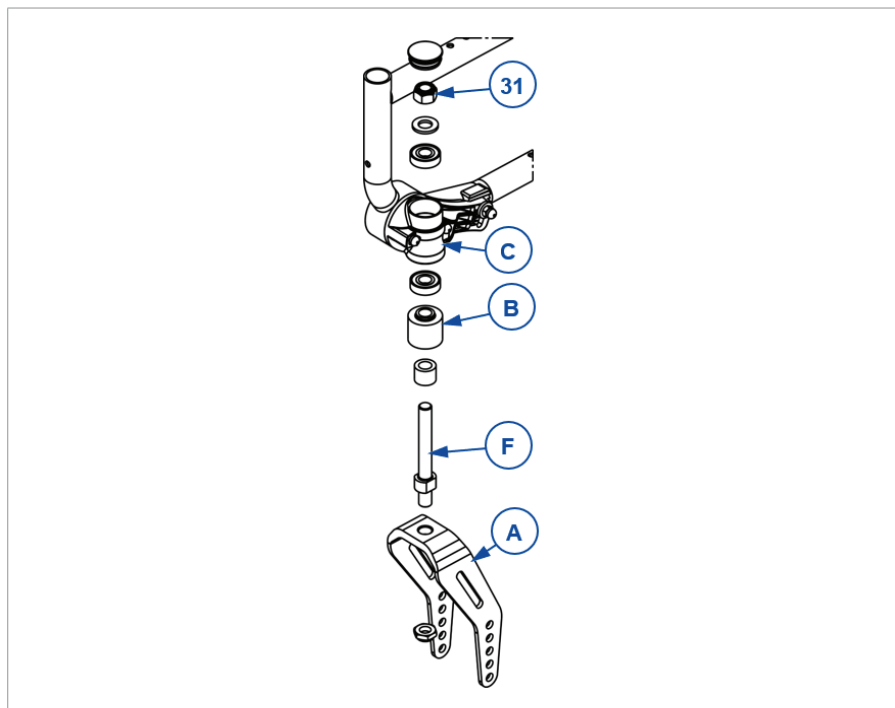


Adjust the front wheel height as follows:

1. Remove the cover cap from the frame tube.
2. Remove the nut (31) of the swivel axle (F) and collect the loose parts.
3. Demount the front wheel and remount it in the correct hole of front fork (A) according to the tables in §3.3.1.
4. Verify that the location of the bushing (B) is correct in relation to the swivel axle housing (C), see tables in §3.3.1.
5. Mount the swivel axle (F) back in the housing (C) with bushing (B) at the correct position, see the exploded view. Tighten nut (31) firmly.
6. Check the tension of the swivel axle (F); make sure it turns smoothly, without play.
7. Repeat this for the second front wheel. Make sure both front wheels are positioned identically.

Final adjustment:

1. Insert the rear wheels.
2. If installed properly, the swivel axles of the front wheels stand perpendicular to the ground. To adjust the angle of the swivel axles, follow the instructions in the next section.
3. Finally, adjust the brakes according to the instructions in §3.6.

3.3.3. Adjusting swivel axles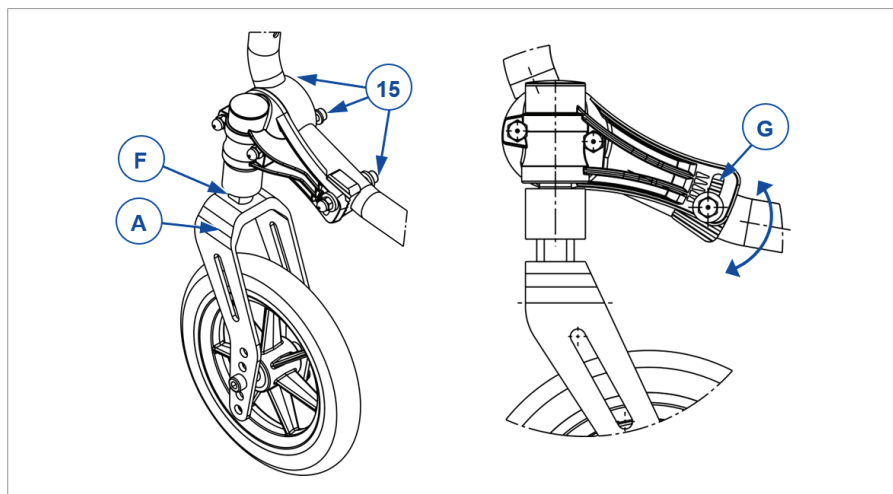

Verify that the swivel axle (F) stands perpendicular to the ground. If not, adjust the swivel axle at toothed clamp (G) as follows:

1. Loosen the three socket screws (15).
2. Adjust the angle of the swivel axle until it is perpendicular to the ground. There are 5 positions available, each indicated by a small triangle.
3. Tighten the 3 socket screws (15) firmly and repeat this for the second swivel axle.

3.4. Seat height

i Applicable to: V100 models


The seat of the V100 wheelchairs is adjustable in 3 seat heights. At each height, there is a different setting of the front and rear wheels.

⚠ WARNING

Risk of injuries


- The top hole of the axle plate is not intended for mounting the rear axle; this hole is reserved for mounting the drum brake.

Seat height	Rear wheels	Front wheels
470 mm	Hole 1	Both bushings above the castor stem housing
500 mm (Default)	Hole 2	One bushings above, and one under the castor stem housing
530 mm	Hole 3	Both bushings under the castor stem housing

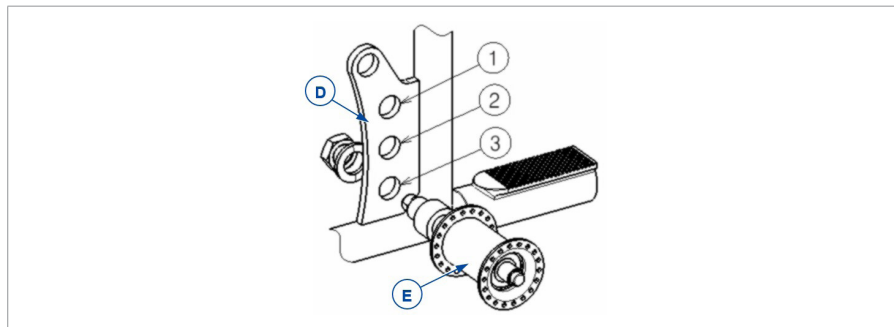Adjust the rear wheel height as follows:

1. Remove the rear wheels (see instruction manual).
2. Demount the axle housings (E) of the rear wheels from the axle plate (D) and remount in the correct hole, see table above.
3. Repeat this for the second rear wheel. Make sure both housings are positioned identically.
4. Verify that both axle housings (E) are tightened firmly.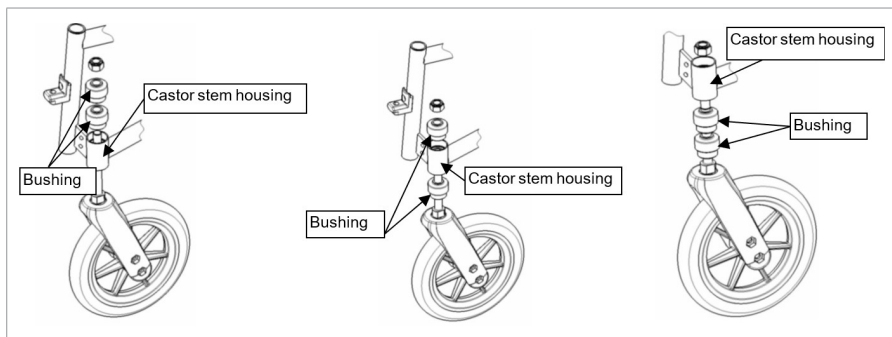

Adjust the front wheel height as follows:

1. Unscrew the swivel axles of the front wheels.
2. Place the bushings according to the table above.
3. Install the swivel axles of the front wheels and tighten. Check the tension in the swivel axle so it turns smoothly, but there is no space for motion.


Final adjustment:

1. Insert the rear wheels.
2. If installed properly, the swivel axles of the front wheels are perpendicular to the ground.
3. Finally, adjust the brakes according to the instructions in **§3.6**.

3.5. Seat depth

The procedure of this adjustment is explained in §3.5.7.


The adjustment settings depend on the wheelchair model as shown in the tables in sections §3.5.1., §3.5.2., §3.5.3. and §3.5.4.. Each table is followed by an illustration of the adjustment points.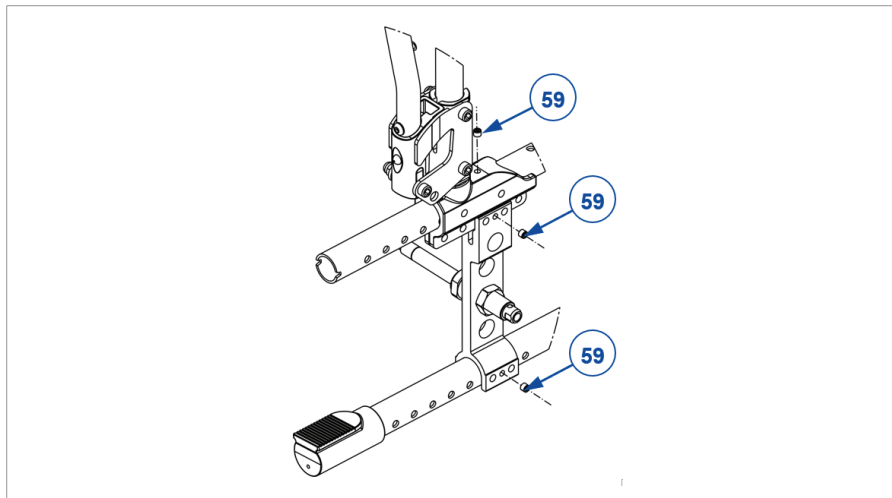

i Important remark

When adjusting the backrest connection, make sure that all 3 connection screws (item 59, in example) are tightened firmly. To prevent that theft-protection systems in shops react on the wheelchair frame, the connection screws (59) must be turned through the frame coating!


3.5.1. Seat depth option 1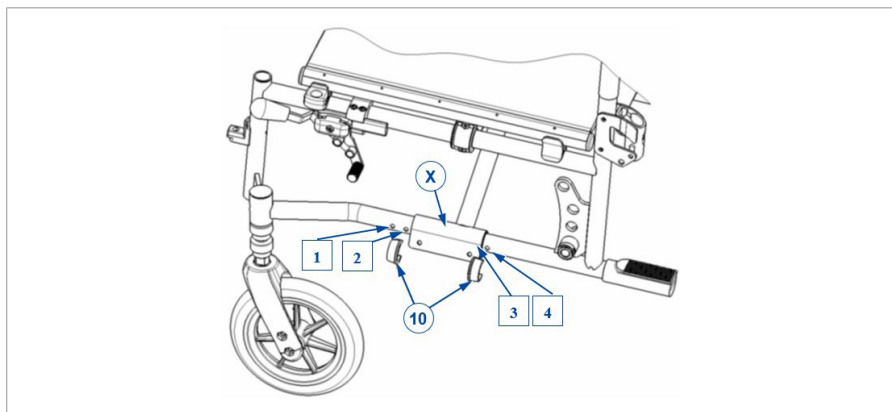

This type of wheelchair is adjustable in 2 seat depths by adjusting the location of the frame cross.

V100	Seat depth	Cross
	450 mm	Hole 1 and 3
	430 mm (Default)	Hole 2 and 4


3.5.2. Seat depth option 2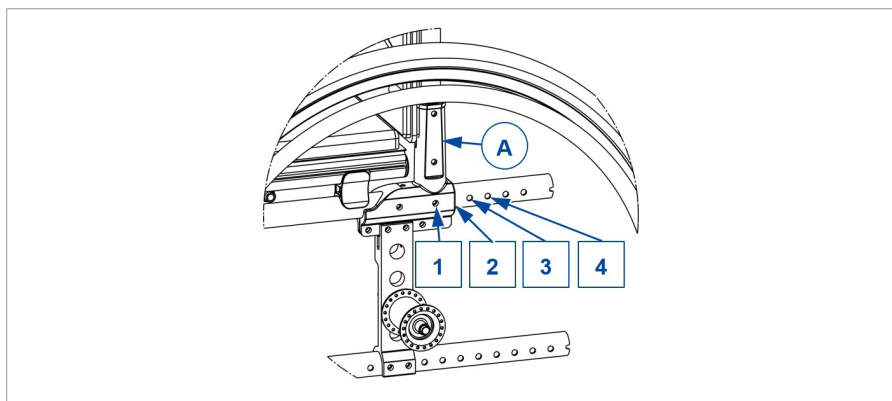

This type of wheelchair is adjustable in 4 seat depths by adjusting the backrest connection (A).

V300 XL	Seat depth	Backrest position holes
	490 mm (Default)	1
	510 mm	2
	530 mm	3
	550 mm	4


3.5.3. Seat depth option 3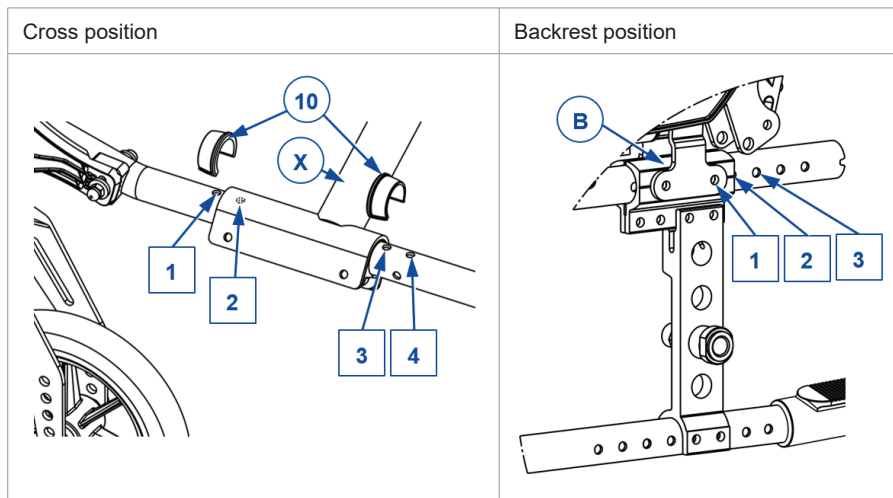

This type of wheelchair is adjustable in 5 seat depths by adjusting the location of the frame cross (X) and backrest connection (B).

V300 30°	Seat depth	Cross position holes	Backrest position holes
	460 mm	Hole 1 and 3	1
	440 mm (Default)	Hole 2 and 4	1
	500 mm	Hole 1 and 3	2
	480 mm	Hole 2 and 4	2
	520 mm	Hole 1 and 3	3


3.5.4. Seat depth option 4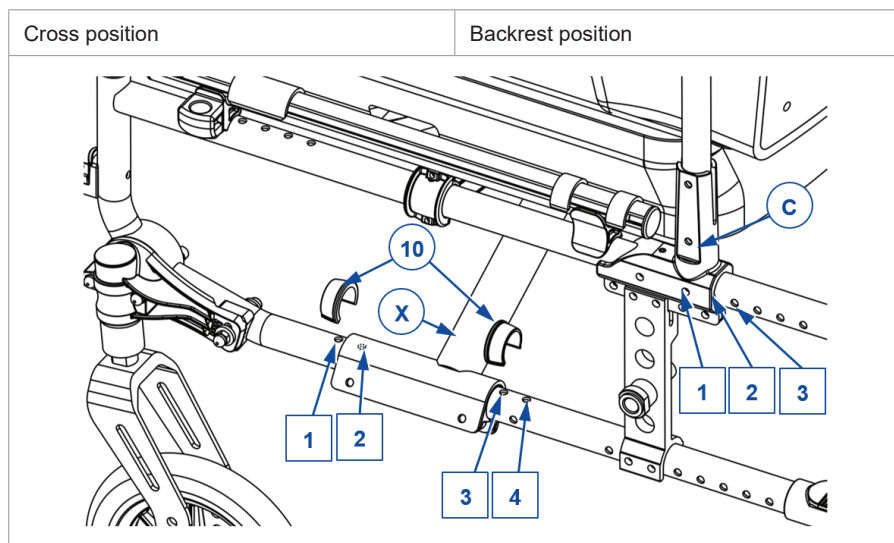

This type of wheelchair is adjustable in 5 seat depths by adjusting the location of the frame cross (X) and backrest connection (C).

V300 V300DC V300F V300ACT V300XR V300DL	Seat depth	Cross position holes	Backrest position holes
	460 mm	Hole 1 and 3	1
	440 mm (Default)	Hole 2 and 4	1
	500 mm	Hole 1 and 3	2
	480 mm	Hole 2 and 4	2
	520 mm	Hole 1 and 3	3


3.5.5. Seat depth option 5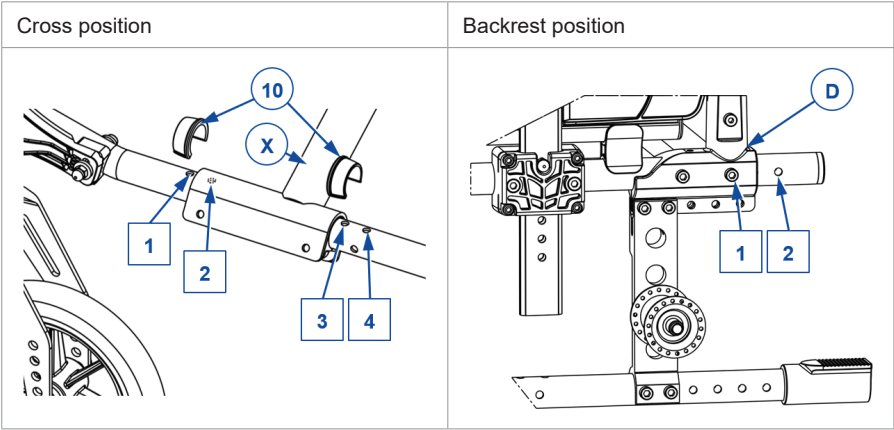

This type of wheelchair is adjustable in 4 seat depths by adjusting the location of the frame cross (X) and backrest connection (D).

V500	Seat depth	Cross position holes	Backrest position holes
	460 mm	Hole 1 and 3	1
	440 mm (Default)	Hole 2 and 4	1
	500 mm	Hole 1 and 3	2
	480 mm	Hole 2 and 4	2

3.5.6. Seat depth option 6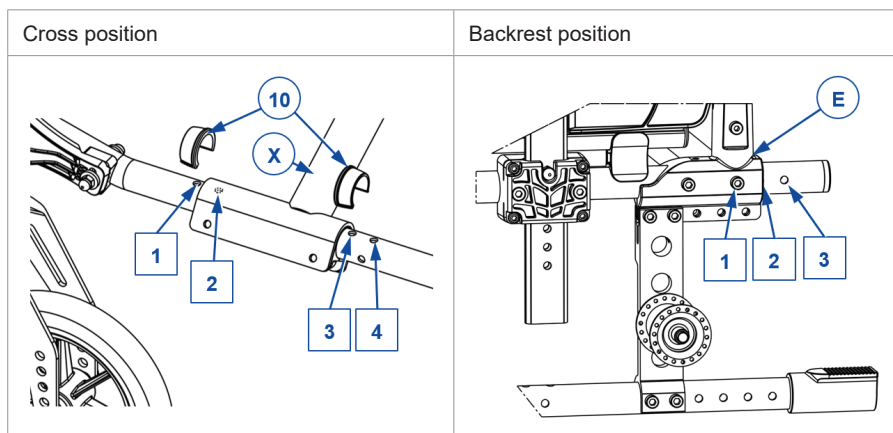

This type of wheelchair is adjustable in 4 seat depths by adjusting the location of the frame cross (X) and backrest connection (E).

V500 Light V500 Light 30° V500 Active	Seat depth	Cross position holes	Backrest position holes
	440 mm	Hole 1 and 3	1
	420 mm (Default)	Hole 2 and 4	1
	480 mm	Hole 1 and 3	3
	460 mm	Hole 1 and 3	2

3.5.7. Adjustment of cross and backrest connection

i The settings and item numbers can be found in previous sections

3.5.7.1. Cross position (if adjustable)

1. Remove clip (10), at each side of the wheelchair.
2. Move the cross (X) forwards/backwards to the desired position according to the settings in the tables of previous sections.
3. Replace both clips (10) at each side of the wheelchair to lock the cross position.

3.5.7.2. Backrest connection position

1. Demount both socket screws of the backrest connection, at both sides of the wheelchair.
2. Move the backrest connection (A/B/C/D/E) forwards/backwards to the desired position according to the settings in the tables of previous sections.
3. Mount the 2 socket screws firmly; at both sides of the wheelchair.
4. Verify if seat height and seat angle are correctly set.

3.5.7.3. Armrest attachment position

Adjust the position of the front armrest receiver according to the backrest connection position.

3.5.7.4. Brake position



Adjust the brake position according to the backrest connection position.

3.6. Parking brakes

⚠ CAUTION

Risk of injuries


- Always adjust the parking brakes after adjusting the rear wheel position. Readjust also when tyres are worn, to compensate the wear.

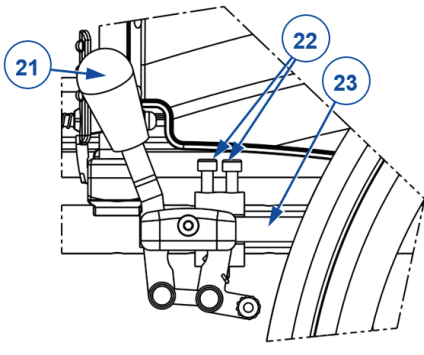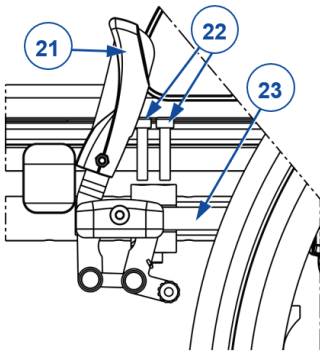	
V100 V100F V100 XL V300 V300 30° V300 XL V300ACT V300DC V300F V300XR	V300DL V500 V500 Light V500 Light 30°

Adjust the correct braking force.


1. Release the brake with lever (21), see instruction manual.
2. Loosen socket screws (22) to make the brake free to slide.
3. Slide the brake over the guide (23) to the desired position.
4. Retighten the socket screws (22).
5. Verify if the brake is working properly: the wheel is blocked, but the brake can be operated easily. If this is not the case, repeat the steps above.
6. Repeat this adjustment for the second parking brake. Make sure both brakes are adjusted identically.

3.6.1. Sport brakes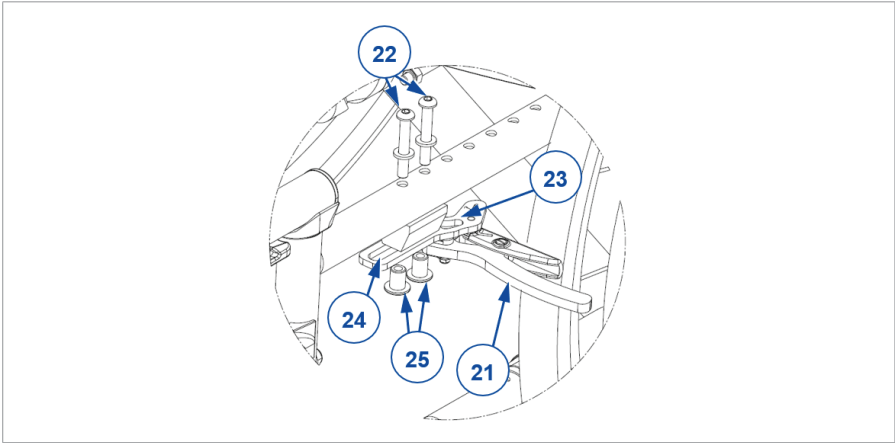

1. Release the brake with lever (21), see instruction manual.
2. If not possible to adjust the brake properly using the slot (24), remove the socket screws (22) and nuts (25).
3. Move the brake (23) to the desired position (6 possible positions).
4. Retighten the socket screws and nuts (22+25) and tighten slightly.
5. Make final adjustments to the brake position by sliding the brake along the slot (24) in the attachment plate.
6. Verify if the brake is working properly: the wheel is blocked, but the brake can be operated easily. If this is not the case, repeat the steps above.
7. Repeat this adjustment for the second parking brake. Make sure both brakes are adjusted identically.

3.7. Footrest length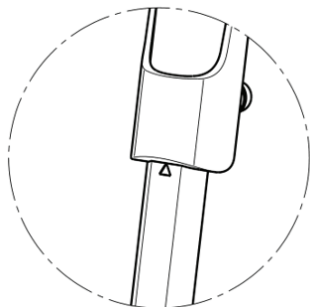

⚠ CAUTION

Risk of damage

- Make sure that the footrests won't touch the ground. Keep a minimum safe distance of 60 mm above the ground.
- The maximum length of the footrest is limited to the length indicated by a triangle on the front of the footrest tube, see picture.

i Since they are connected, both footrests of V300XL are adjusted simultaneously.


3.7.1. Stepless adjustment

V100 V300 - V500	V100 XL V300 XL

Adjust the length as follows:

1. Remove the socket screw (26) at the back of the footrest.
2. Adjust the footrest to a comfortable length.
3. Retighten the socket screw firmly.
4. Repeat this for the second footrest. Make sure the footrests are positioned identically.


3.7.2. Adjustment in steps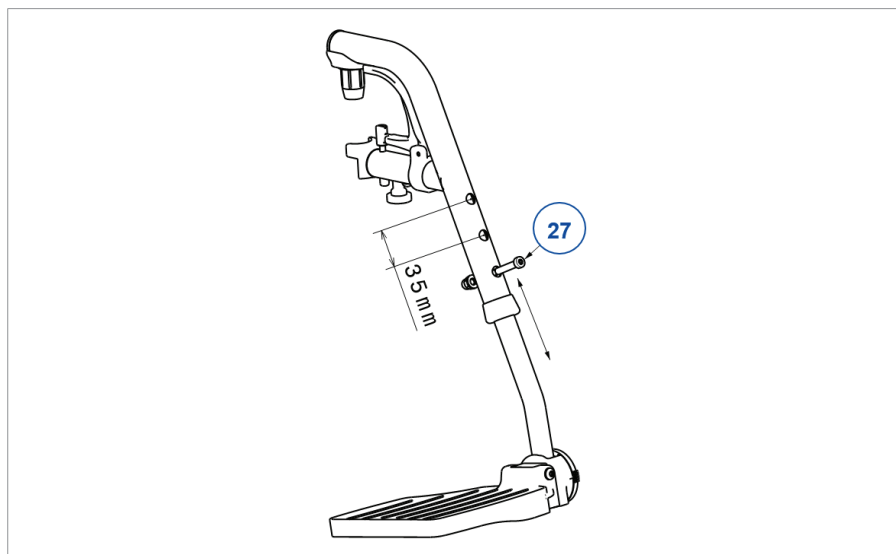

Adjust the length as follows (3 holes on inside tube, 3 holes on outside tube ; 120mm range):

1. Remove the screw (27).
2. Adjust the footrest to a comfortable length.
3. Retighten the screw firmly.
4. Repeat this for the second footrest. Make sure the footrests are positioned identically.

3.8. Footplate position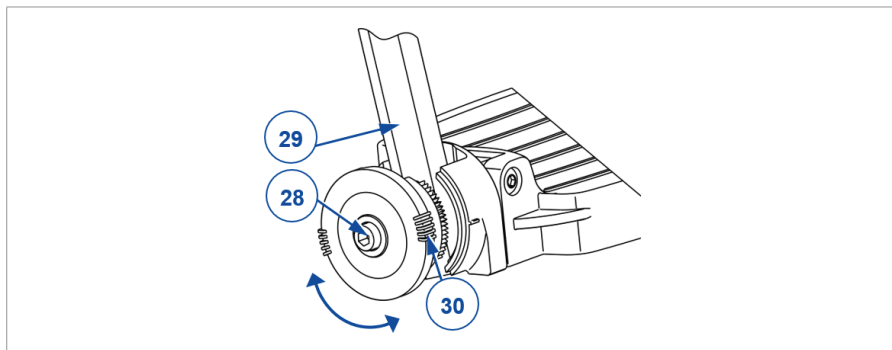

The footplate(s) can be used in 2 positions: front and rear. By default, each footplate is adjusted with its base plate in the rear position.

To place the footplate in front position:


1. Remove the socket screws (28) of both footplates.
2. Exchange left and right footplate.
3. Tighten the socket screws (28) firmly.

3.9. Footplate angle

Adjust the angle of the footplate at its toothed clamp:

1. Loosen the socket screw (28) of the footrest (29).
2. Turn the footplate to the desired angle (80° - 100°). The dashes (30) on the toothed clamp indicate the angular positions.
3. Retighten the socket screw.
4. Repeat this for the second footplate.

3.10. Arm pad height and depth (option 1)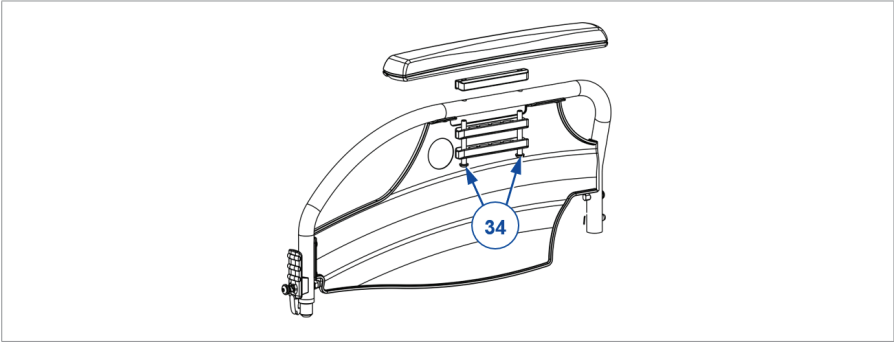

This arm pad is adjustable in height and depth, each in 3 positions. The height is adjusted by adding or removing distance blocks. The depth is adjusted by using different mounting holes.

To adjust:

1. Remove both socket screws (34).
2. Place the correct number of blocks below and above the frame tubes, see table 1.
3. For depth adjustment, install the arm pad by using the correct mounting holes, see table 2.


Table 1

Height arm pad to seat	Number of blocks below tube	Number of blocks between tube and arm pad
220 mm (Default)	2	1
230 mm	1	2
240 mm	0	3

Table 2

Depth of the arm pad	Used holes
Front position	Back and middle
Middle position (Standard)	Hole 2 and 4
Back position	Front and middle


3.11. Arm pad depth (option 2)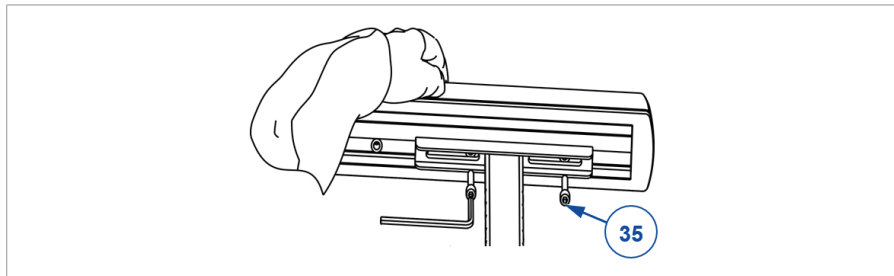

Adjust the depth of the arm pad as follow (range 120 mm: step less):

1. Loosen both socket screws (35) at the bottom of the arm pad.
2. Move the arm pad to the front or rear within the range of the slotted holes (42,5 mm).
If this not enough, remove the socket screws and attach the arm pad using the other holes (3 positions available).
3. Tighten both socket screws firmly.
4. Repeat this for the second arm pad. Make sure both arm pads are positioned identically.


3.12. Armrest height (option 3)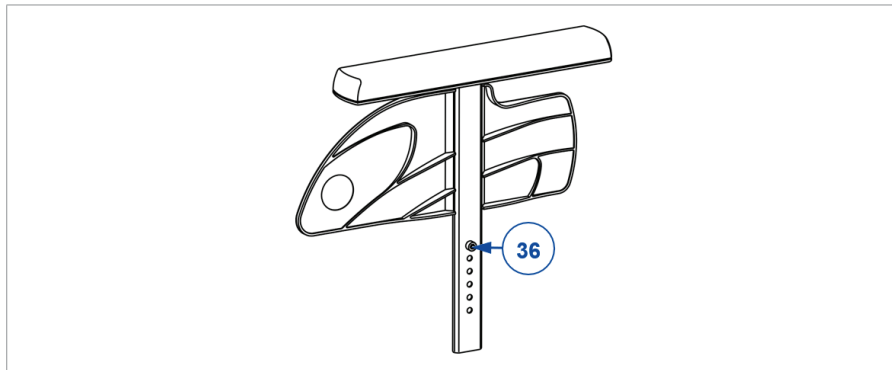

The height of the armrest is adjustable in 6 positions as follows:

1. Pull the armrest out of the armrest holder.
2. Remove the socket screw (36) that sets the height.
3. Mount the screw in the desired hole.
4. Insert the armrest in the armrest holder.
5. Check the height; readjust if necessary.
6. Repeat this for the second armrest. Make sure both armrests are positioned identically.

3.13. Side plate adjustment (option 4)

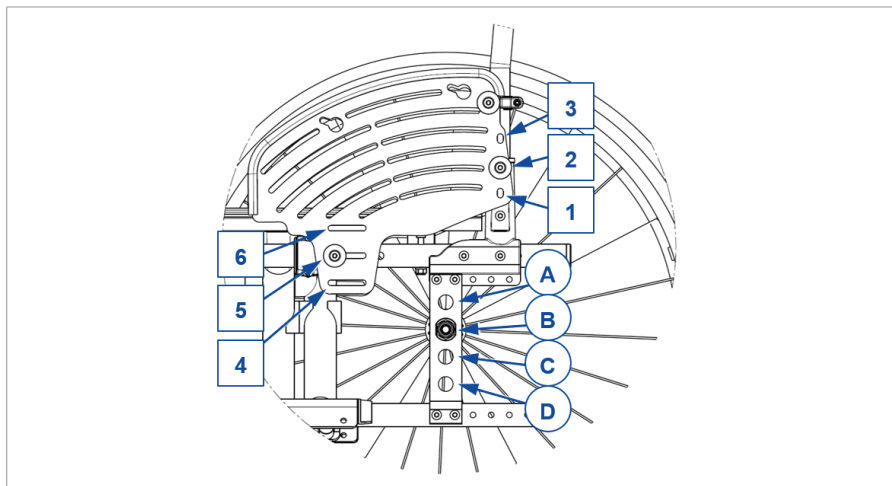The height of the side plate can be adjusted in 3 positions as follows:

1. Remove socket screws (backrest screw & seat screw).
2. Move the side plate up/down to the desired height and retighten it with the socket screws in the corresponding openings according to the table below.

Repeat the adjustments above for the second side plate. Make sure that both side plates are set to the same height and dept, and that they are firmly secured.

Wheel position	Backrest screw	Seat screw
A	1	4
B	2	5
C	3	6
D	3	6

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All rights reserved, including translation.