


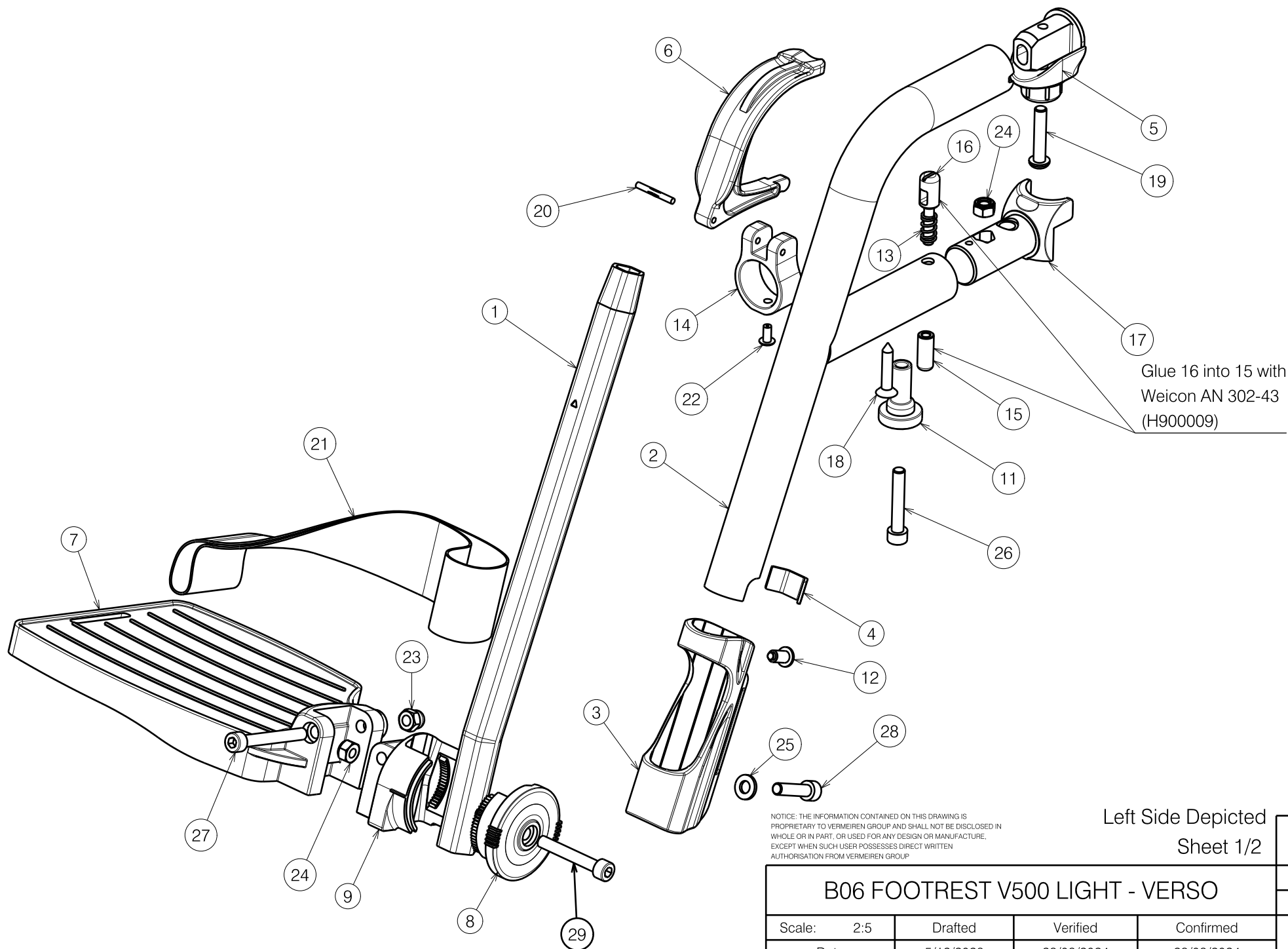

Left Side Depicted
Sheet 1/2

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN
AUTHORISATION FROM VERMEIREN GROUP

B06 FOOTREST V500 LIGHT - VERSO				Plan No.
				E00071954/E.1
Scale: 2:5	Drafted	Verified	Confirmed	
Date	5/12/2023	28/08/2024	28/08/2024	
Initials	SD	DVB	PJ	

Material			
No.	Plan	Article	Qty
1	P00048568	20855	1
2	P00048569	20876..	1
3	P00082368	3994782	1
4	Y016501	1733500	1
5	X004414	1904130	1
6	X004418	1904207	1
7A	X004570	1904646 (L)	1
	X004571	1904647 (R)	1
7B	X004457	1904475 (L)	1
	X004453	1904210 (R)	1
7C	X004572	1904648 (L)	1
	X004573	1904649 (R)	1
8	X004464	1904212	1
9	X004465	1904213	1
11	X004660	1905195	1
12	X004687	1905639	1
13	XC00142	3990576	1
14	XC00300	3991091	1

Material			
No.	Plan	Article	Qty.
15	XC00304	3991093	1
16	XC00305	3991094	1
17	XC00381	3991544	1
18	S-Point 5x25	H016007	1
19	M6x30 ISO7380	1009588	1
20	D3x24 DIN1475	1905640	1
21	---	1905642	1
22	D4x8 DIN7337A	1992805	1
23	M6 DIN986	2994059	1
24	M6 DIN985	2994063	2
25	M6 DIN125	2994414	1
26	M6x35 DIN912	2994634	1
27	M6x50 DIN912	2995258	1
28	M6x25 DIN912	2996887	1
29	M6x40 DIN912	2994317	1

Quantities are for one side

7A sizes 39-42

7B sizes 44-46

7C sizes 48-50

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN
AUTHORISATION FROM VERMEIREN GROUP

Sheet 2/2

Plan No.

E00071954/E.1

B06 FOOTREST V500 LIGHT - VERSO			
Scale: 2:5	Drafted	Verified	Confirmed
Date	5/12/2023	28/08/2024	28/08/2024
Initials	SD	SD	PJ

