


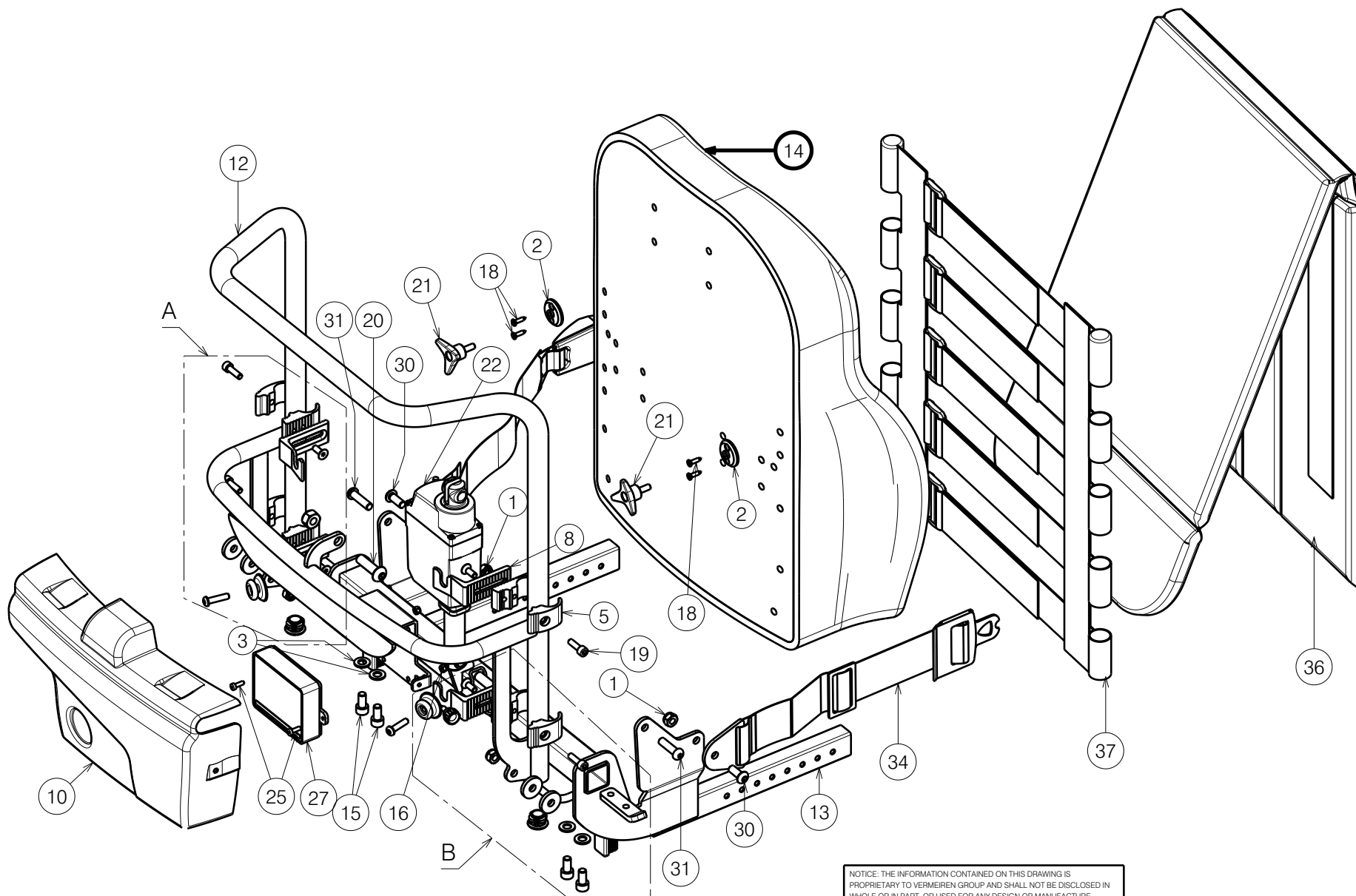

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

Sheet 1/4



SIGMA BACK FRAME L34 TUB /L44 SE41

Plan No.

E001104/K

Scale: 1:6	Drafted	Verified	Confirmed
Date	20/06/2018	1/06/2026	1/06/2026
Initials	SD	PM	MK



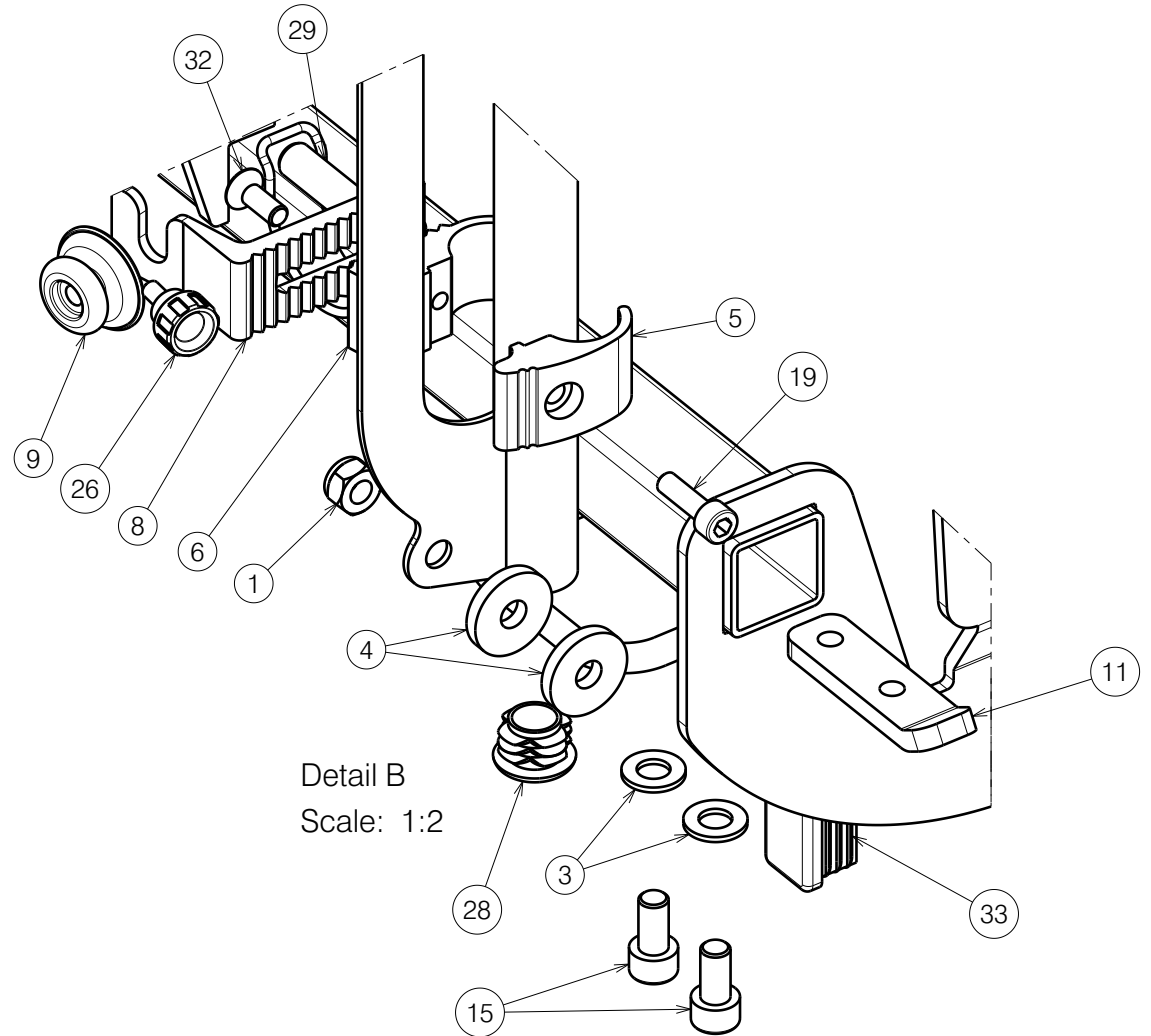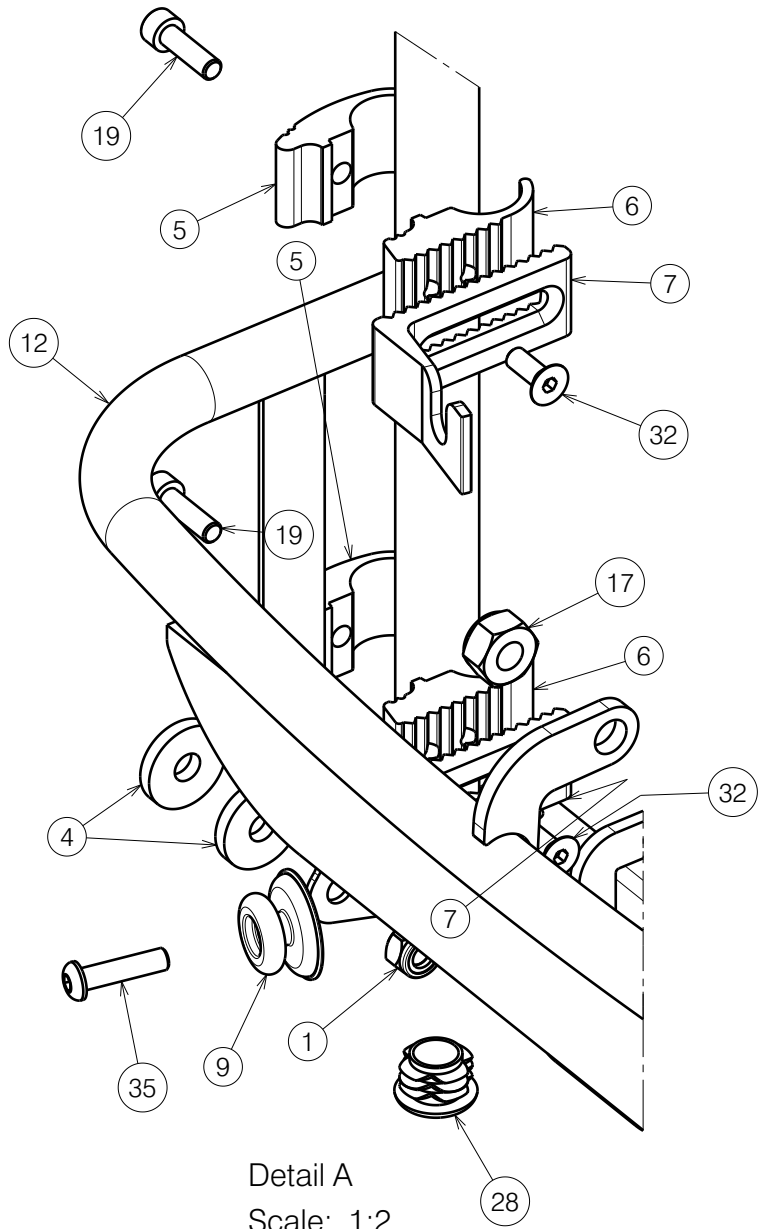

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

Sheet 2/4

SIGMA BACK FRAME L34 TUB /L44 SE41

Plan No.

E001104/K

Scale: 1:6

Drafted

Verified

Confirmed

Date

20/06/2018

1/06/2026

1/06/2026

Initials

SD

PM

MK



Material				
No.	Plan	Article	Quantity	
1	X001016	1001682	4	
2	X001052	G066600	2	
3	X001119	1001554	4	
4	X004048	1902685	4	
5	X004820	1744599	4	
6	X004821	1744699	4	
7	X004822	1744799	2	
8	X004823	1744899	2	
9	X004861	1906454	2	
10	X005410	1914781	1	
11	Y018564	1774200	2	
12	YP09205	2638854 (40)	1	
	P00078007	2686754 (42)	1	
	YP09206	2638954 (45)	1	
	P00078470	2686854 (48)	1	
	YP09207	2639054 (50)	1	
	13	Y022312	2049854 (40)	1
		P00077920	2686354 (42)	1
Y022980		2049754 (45)	1	
P00077989		2686454 (48)	1	
14	Y022313	2049954 (50)	1	
	Soft grey 40	1912001	1	
	Soft grey 42	2998402	1	
	Soft grey 45	1911493	1	
	Soft grey 48	2998407	1	
	Soft grey 50	1924046	1	
	Soft black 40	2993418	1	
	Soft black 42	2998408	1	
	Soft black 45	2993419	1	
	Soft black 48	2998409	1	
Soft black 50	2993420	1		
15	M8x16 DIN912	1001498	4	
16	M4 DIN985	1001678	2	
17	M10 DIN985	1001683	1	
18	D3,9x16 DIN7982	1005767	4	
19	M6x20 DIN912	1001493	4	
20	M10x35 ISO7380	1009442	1	

Material			
No.	Plan	Article	Quantity
21	M6x15	1009932	2
22	LA23	1906790	1
23	---	1906791	1
24	---	1906792	1
25	M4x12 DIN7984	1907582	2
26	M5x10	1911896	1
27	----	1912307	1
28	D22	1990648	2
29	D10x40	1991481	1
30	M8x20 ISO7380	1992349	2
31	M8x30 ISO7380	1992717	2
32	M6x16 DIN7991	1994752	4
33	30x20	1997922	2
34	---	2993352	1
35	M6x25 ISO7380	1006998	2
36	P00082952	2998324 (40)	1
	P00083522	2998446 (42)	1
	X004857	1906740 (45)	1
	P00082953	2998447 (48)	1
	X004889	1906841 (51)	1
	Airflow	2998325 (40)	1
	Airflow	2998448 (42)	1
	Airflow	2995211 (45)	1
	Airflow	2998449 (48)	1
	Airflow	2995212 (51)	1
Cordura	2998326 (40)	1	
Cordura	2998450 (42)	1	
Cordura	2995208 (45)	1	
Cordura	2998451 (48)	1	
Cordura	2995209 (50)	1	

Material			
No.	Plan	Article	Quantity
37	A00082037	1906843(40)	1
		2998461(42)	1
		1906844(45)	1
		2998462(48)	1
		1906845(50)	1

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP

Sheet 3/4

SIGMA BACK FRAME L34 TUB /L44 SE41

Scale: 1:6

Drafted

Verified

Confirmed

Date

20/06/2018

1/06/2026

1/06/2026

Initials

SD

PM


MK

Plan No.

E001104/K



*Option.

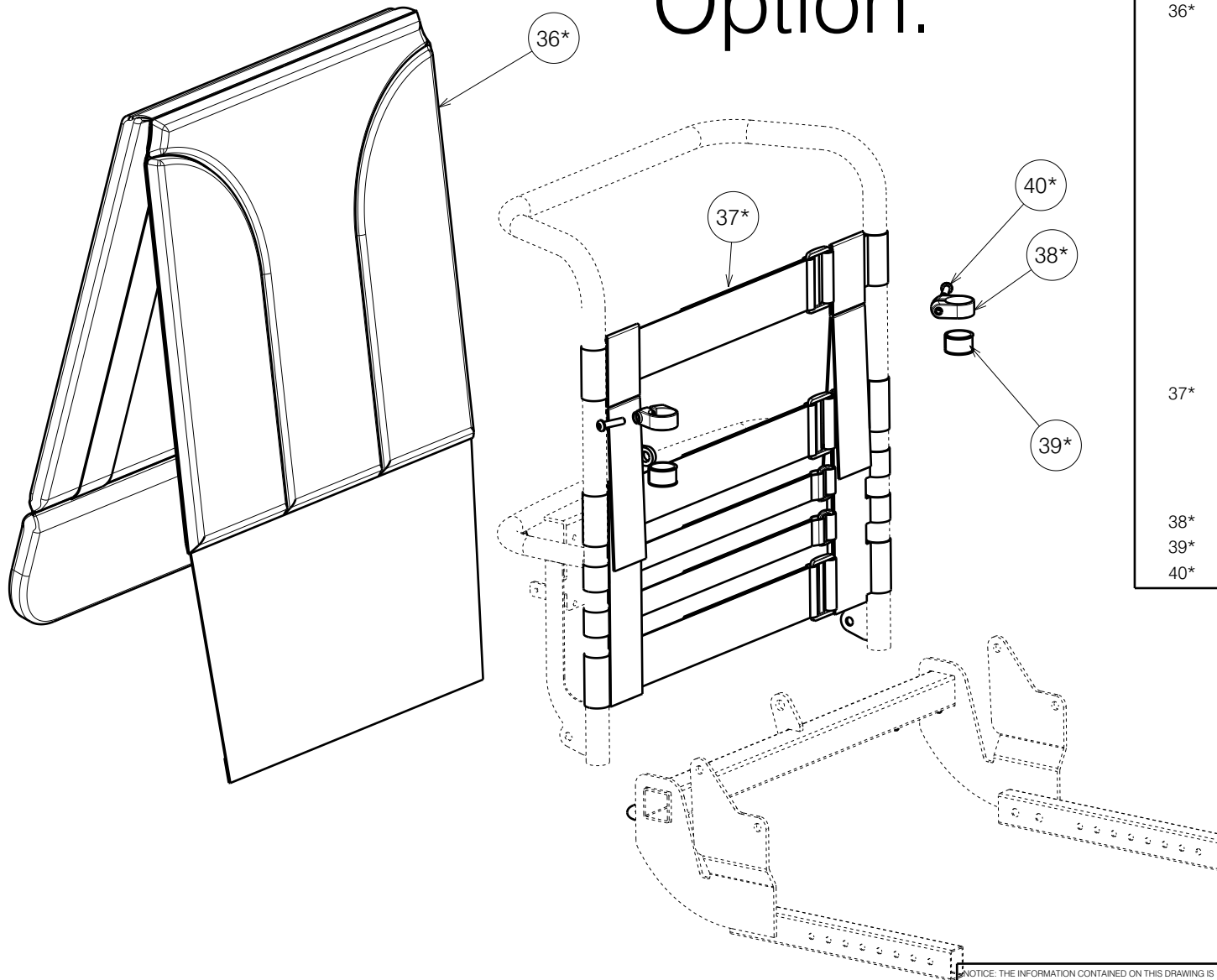

L44 WITH HEIGHT ADJUSTMENT			
No.	Plan	Article	Quantity
36*	P00079403	2998410(40)	1
	P00079237	2998411(42)	1
	P00079573	2998412(45)	1
	P00079238	2998413(48)	1
	P00079239	2998414(51)	1
	Airflow	2998467 (40)	1
	Airflow	2998426 (42)	1
	Airflow	2998427 (45)	1
	Airflow	2998428 (48)	1
	Airflow	2998429 (51)	1
	Cordura	2998468 (40)	1
	Cordura	2998432 (42)	1
	Cordura	2998433 (45)	1
	Cordura	2998434 (48)	1
	Cordura	2998435 (51)	1
37*	A00079244	2998415(40)	1
	---	2998416(42)	1
	---	2998417(45)	1
	---	2998418(48)	1
	---	2998419(51)	1
38*	XP01976	2993641	2
39*	XP01980	2993813	2
40*	M6x25	2994062	2

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

Sheet 4/4

SIGMA BACK FRAME L34 TUB /L44 SE41

Plan No.

E001104/K

Scale: 1:6

Drafted

Verified

Confirmed

Date

20/06/2018

1/06/2026

1/06/2026

Initials

SD

PM

MK

