

U69 RIGHT GEMINI LIGHT GREY: 2996814			
No.	Plan	Article	Quantity
1	FABRIC ALCALA (22) LIGHT GREY	2996799	0,1m
2	XP03345	2997118	1
3	YP04585	28691	1

U69 LEFT GEMINI LIGHT GREY: 2996815			
No.	Plan	Article	Quantity
1	FABRIC ALCALA (22) LIGHT GREY	2996799	0,1m
2	XP03345	2997118	1
3	YP04586	28692	1

U69 RIGHT GEMINI BLACK/BLUE FAB/MESH: 2996816			
No.	Plan	Article	Quantity
1	FABRIC ALCALA (14) BLACK	2991844	0,06m
2	XP03345	2997118	1
3	YP04585	28691	1
4	FABRIC AIR FLOW BLUE 9570	2992088	0,06m


U69 LEFT GEMINI BLACK/BLUE FAB/MESH: 2996817			
No.	Plan	Article	Quantity
1	FABRIC ALCALA (14) BLACK	2991844	0,06m
2	XP03345	2997118	1
3	YP04586	28692	1
4	FABRIC AIR FLOW BLUE 9570	2992088	0,06m

U69 BOLTS AND NUTS - quantities for both sides: 2995357			
No.	Plan	Article	Quantity
5	Rivet 4x14	1001584	4
6	XP00223	2990144	4

2995354 - set for all U69 bolts and nuts

U69 SIDE TRUNK PELOTS WITHOUT ADJUSTMENT GEMINI 2020				Plan No.
Page 1/2				EP00617/ C
Scale: 1:10	Drafted	Verified	Confirmed	
Date	21/09/2021	15/02/2022	15/02/2022	
Initials	OY	AZ	TK	

EP00617 / C

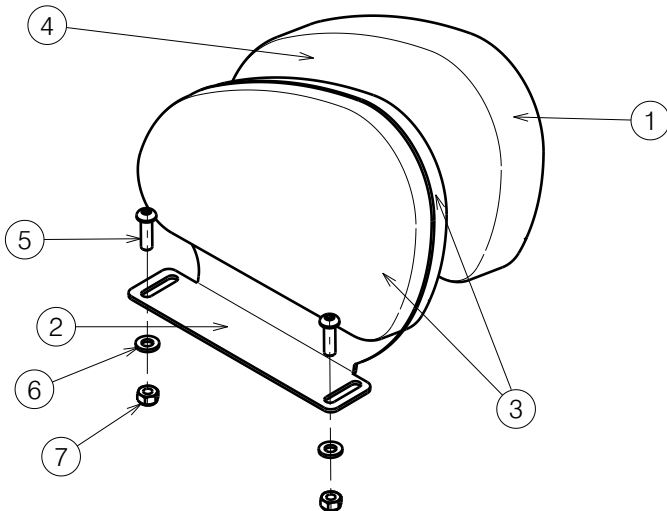

EP00617/ C

SEAT PELOT BIG R LIGHT GREY GEMINI 32: 2996837			
No.	Plan	Article	Quantity
1	FABRIC ALCALA (22) LIGHT GREY	2996799	0,12m
2	YP04551	28689	1
3	XP03346	2997119	1

SEAT PELOT BIG L LIGHT GREY GEMINI 32: 2996838			
No.	Plan	Article	Quantity
1	FABRIC ALCALA (22) LIGHT GREY	2996799	0,12m
2	YP04552	28690	1
3	XP03346	2997119	1

SEAT PELOT BIG R BLACK/BLUE FAB/MESH GEMINI 32: 2996839			
No.	Plan	Article	Quantity
1	FABRIC ALCALA (14) BLACK	2991844	0,08m
2	YP04551	28689	1
3	XP03346	2997119	1
4	FABRIC AIR FLOW BLUE 9570	2992088	0,08m

SEAT PELOT BIG L BLACK/BLUE FAB/MESH GEMINI 32: 2996840			
No.	Plan	Article	Quantity
1	FABRIC ALCALA (14) BLACK	2991844	0,08m
2	YP04552	28690	1
3	XP03346	2997119	1
4	FABRIC AIR FLOW BLUE 9570	2992088	0,08m

SEAT PELOT BIG R LIGHT GREY GEMINI 40: 2996990			
No.	Plan	Article	Quantity
1	FABRIC ALCALA (22) LIGHT GREY	2996799	0,14m
2	YP04554	28837	1
3	XP03349	2997122	1

SEAT PELOT BIG L LIGHT GREY GEMINI 40: 2996991			
No.	Plan	Article	Quantity
1	FABRIC ALCALA (22) LIGHT GREY	2996799	0,14m
2	YP04555	28838	1
3	XP03349	2997122	1

SEAT PELOT BIG R BLACK/BLUE FAB/MESH GEMINI 40: 2996992			
No.	Plan	Article	Quantity
1	FABRIC ALCALA (14) BLACK	2991844	0,09m
2	YP04554	28837	1
3	XP03349	2997122	1
4	FABRIC AIR FLOW BLUE 9570	2992088	0,09m

SEAT PELOT BIG L BLACK/BLUE FAB/MESH GEMINI 40: 2996993			
No.	Plan	Article	Quantity
1	FABRIC ALCALA (14) BLACK	2991844	0,09m
2	YP04555	28838	1
3	XP03349	2997122	1
4	FABRIC AIR FLOW BLUE 9570	2992088	0,09m

BOLTS AND NUTS - quantities for both sides: 2995358			
No.	Plan	Article	Quantity
5	M6x20 ISO 7380	2994910	4
6	M6 DIN 985	2994063	4
7	M6 DIN 125 PVC	2994415	4

U69 SIDE TRUNK PELOTS WITHOUT ADJUSTMENT GEMINI 2020				Plan No.
Page 2/2				EP00617/ C
Scale: 1:10	Drafted	Verified	Confirmed	
Date	21/09/2021	15/02/2022	15/02/2022	
Initials	OY	AZ	TK	