
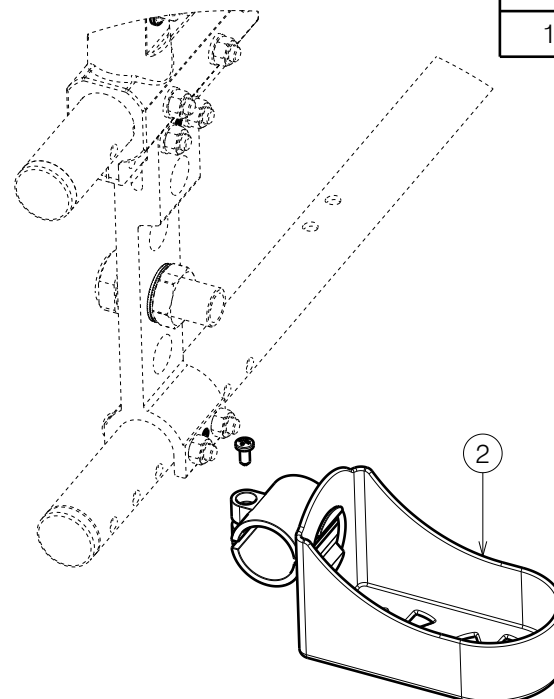


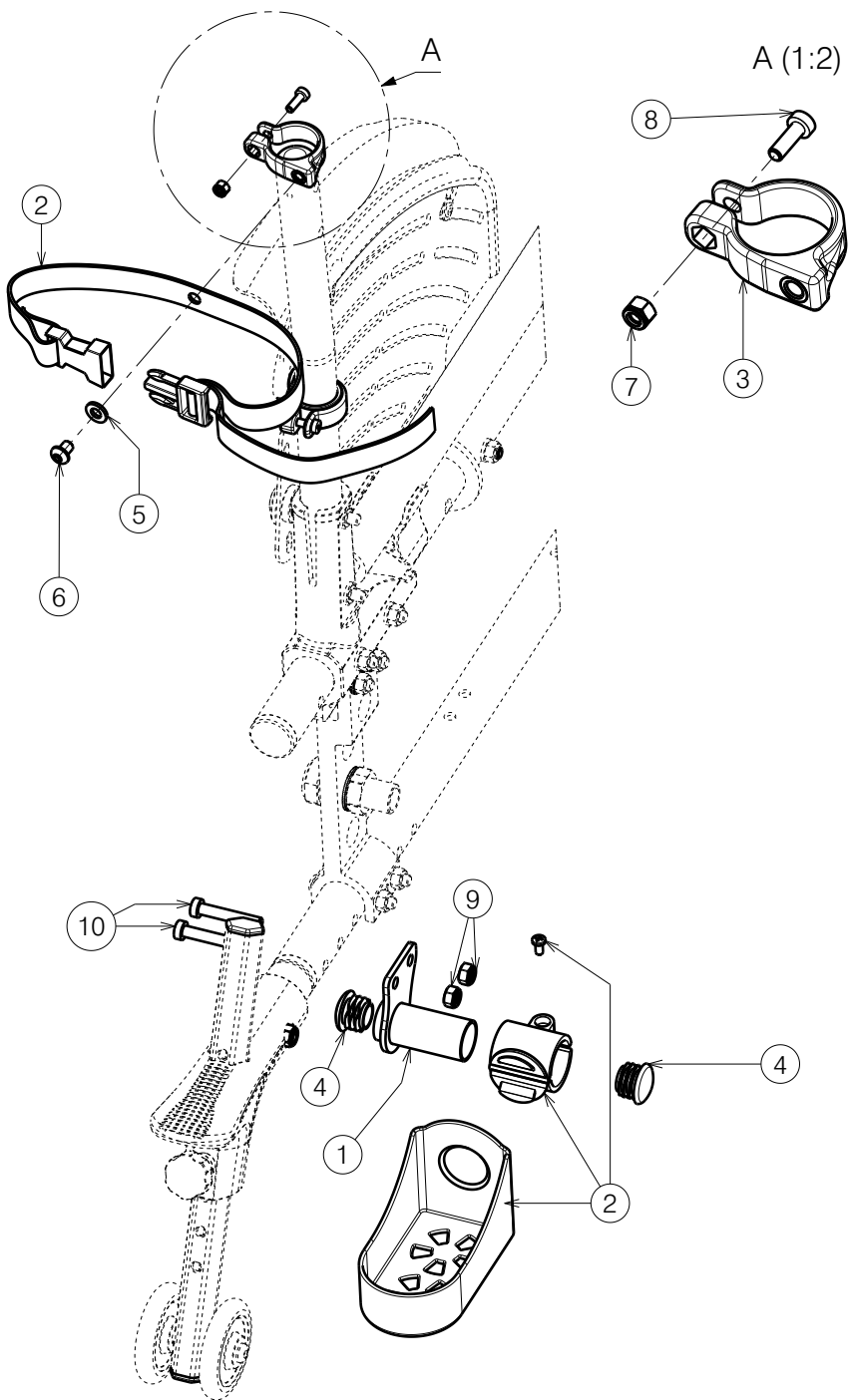

| No. | Plan             | Article | Qty |
|-----|------------------|---------|-----|
| 1   | P00094415(L)     | 2694954 | 1   |
|     | P00094419(R)     | 2694854 | 1   |
| 2   | D22              | 1907511 | 1   |
|     | D25              | 1906616 | 1   |
| 3   | XP02043          | 2993671 | 1   |
| 4   | Cup D22          | 1990648 | 1   |
| 5   | Washer M6        | 2994414 | 1   |
| 6   | M6x8 ISO7380     | 2996034 | 1   |
| 7   | M4 DIN 985 Black | 2994901 | 1   |
| 8   | M4x12 DIN 7984   | 1907582 | 1   |
| 9   | M6 DIN 985       | 2994063 | 2   |
| 10  | M6x35 DIN7984    | 2994654 | 2   |

\* When there is no B78.

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS  
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN  
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,  
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION  
 FROM VERMEIREN GROUP.

## B31 CRUTCH CARRIER V500 ACTIVE

|            |            |            |            |
|------------|------------|------------|------------|
| Scale: 1:4 | Drafted    | Verified   | Confirmed  |
| Date       | 06/12/2024 | 06/12/2024 | 06/12/2024 |
| Initials   | AS         | AZ         | AS         |

Plan No.  
 E00094427/A1.1

