


PRINT IN COLOR

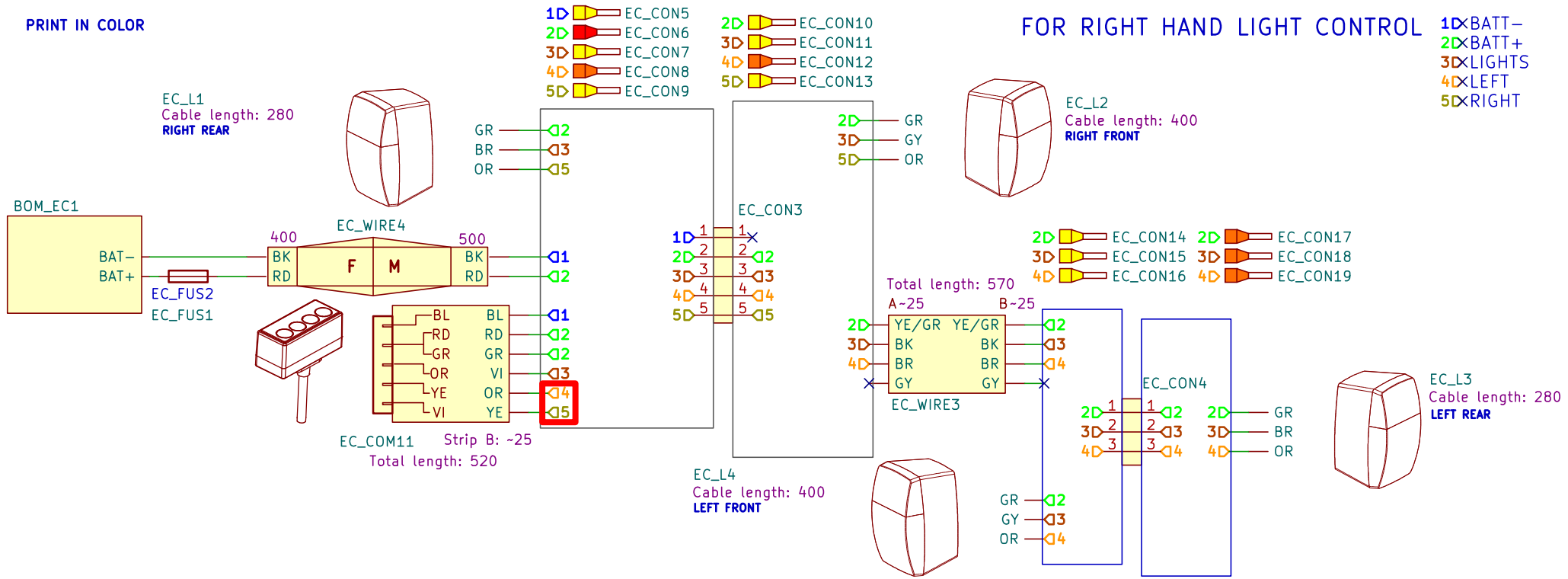

#	Reference	Art number	Description	Qty
1	EC_CON3	2998579	EC CON EW-M20-5P-3-3	1
2	EC_CON4	2998577	EC CON EW-M16-3P-1-2	1
3	EC_CON5,EC_CON7,EC_CON9,EC_CON10,EC_CON11,EC_CON13,EC_CON14,EC_CON15,EC_CON16	2998611	EC CON bootlace ferrules 0,75 mm x0,8 YE	9
4	EC_CON6	2998612	EC CON bootlace ferrules 1 mm x0,8 RD	1
5	EC_CON8,EC_CON12,EC_CON17,EC_CON18,EC_CON19	2998610	EC CON bootlace ferrules 0,5 mm x0,8 OR	5
6	EC_FUS1	2998575	EC FUS FUSE HOLDER 5X20	1
7	EC_L1, EC_L3	1914550	EC L TAIL LIGHT BAYARD AT02L 24V	2
8	EC_L2, EC_L4	1914551	EC L FRONT LIGHT BAYARD AA02L 24V	2
9	EC_COM11	1921014	EC COM LIGHT MODULE	1
		1915936	EC COM LIGHT MODULE SW	1
10	EC_WIRE3	1907749	EC WIRE CABLE 4X0,5 BLACK	0,57M
10	EC_WIRE4	2998580	EC WIRE CABLE M8-M-F-1M	1
12	EC_ZIP2-EC_ZIP11	2993110	EC ZIP BINDING BAND NYLON L=300MM	10
13	EC_FUS2	2998574	EC FUS FUSE 1A QUICK BLOW 5X20	1
14	BOM_EC1	2998597	BOM EC PLEGO VERSO DRIVE TRAIN	1

BK	BLACK
RD	RED
LBL	LIGHT BLUE
BL	BLUE
LGR	LIGHT GREEN
GR	GREEN
VI	VILLET
OR	ORANGE
YE	YELLOW
WH	WHITE
LRD	LIGHT RED
PK	PINK
GY	GRAY
/	AND
	OR

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP

2998566/B

Plan No.
2998566/B

Size: A4

Date:

Initials:

Drafted

2025-02-13

FM

Verified

2025-02-27

FM

Comformed

2025-02-3-27


TK

Id: 1/2

BOM EC VERSO DRIVE TRAIN + SE19 LIGHTS

Id: 1/2

PRINT IN COLOR

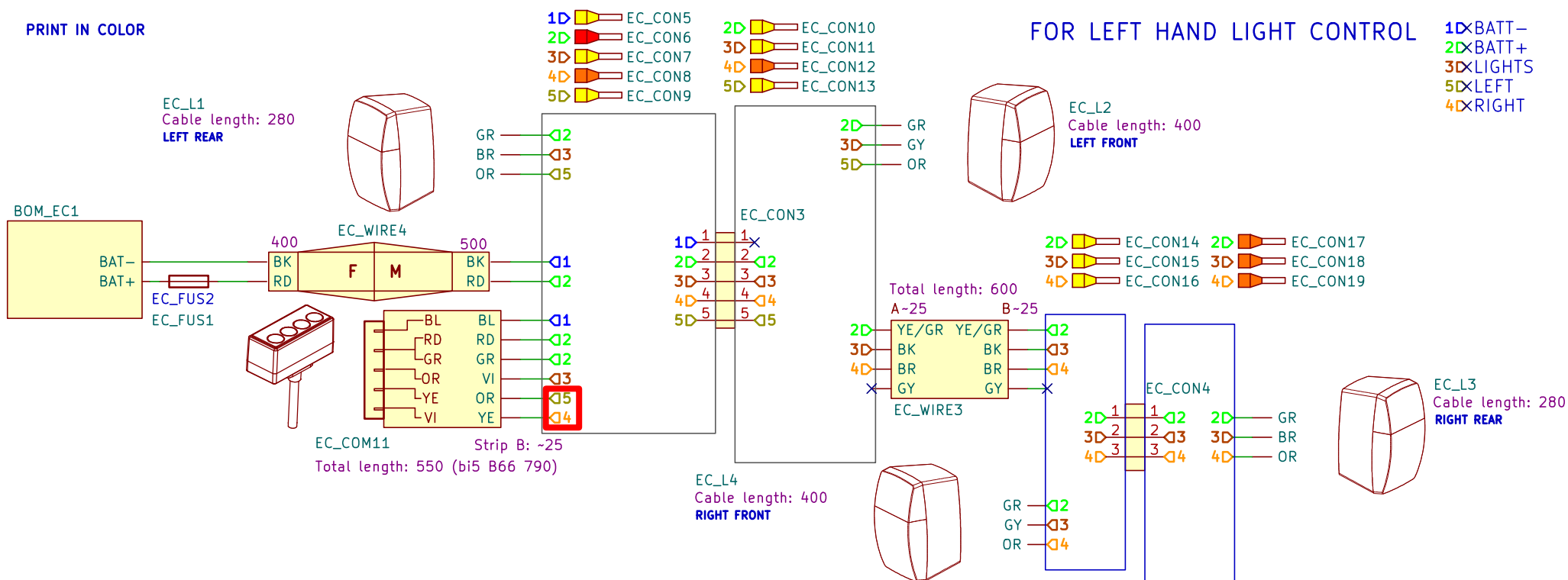

#	Reference	Art number	Description	Qty
1	EC_CON3	2998579	EC CON EW-M20-5P-3-3	1
2	EC_CON4	2998577	EC CON EW-M16-3P-1-2	1
3	EC_CON5,EC_CON7,EC_CON9,EC_CON10,EC_CON11, EC_CON13,EC_CON14,EC_CON15,EC_CON16	2998611	EC CON bootlace ferrules 0,75 mm x0,8 YE	9
4	EC_CON6	2998612	EC CON bootlace ferrules 1 mm x0,8 RD	1
5	EC_CON8,EC_CON12,EC_CON17,EC_CON18,EC_CON19	2998610	EC CON bootlace ferrules 0,5 mm x0,8 OR	2
6	EC_FUS1	2998575	EC FUS FUSE HOLDER 5X20	1
7	EC_L1, EC_L3	1914550	EC L TAIL LIGHT BAYARD AT02L 24V	2
8	EC_L2, EC_L4	1914551	EC L FRONT LIGHT BAYARD AA02L 24V	2
9	EC_COM11	1921014	EC COM LIGHT MODULE	1
		1915936	EC COM LIGHT MODULE SW	0,57M
10	EC_WIRE3	1907749	EC WIRE CABLE 4X0,5 BLACK	1
10	EC_WIRE4	2998580	EC WIRE CABLE M8-M-F-1M	1
12	EC_ZIP2-EC_ZIP11	2993110	EC ZIP BINDING BAND NYLON L=300MM	10
13	EC_FUS2	2998574	EC FUS FUSE 1A QUICK BLOW 5X20	1
14	BOM_EC1	2998597	BOM EC PLEGO VERSO DRIVE TRAIN	1

BK	BLACK
RD	RED
LBL	LIGHT BLUE
BL	BLUE
LGR	LIGHT GREEN
GR	GREEN
VI	VILLET
OR	ORANGE
YE	YELLOW
WH	WHITE
LRD	LIGHT RED
PK	PINK
GY	GRAY
/	AND
	OR

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORIZATION FROM VERMEIREN GROUP

2998566/B

BOM EC VERSO DRIVE TRAIN + SE19 LIGHTS

Id: 2/2

Plan No.

2998566/B

Size: A4	Drafted	Verified	Comformed
Date:	2025-02-27	2025-02-27	2025-02-3-27
Initials:	FM	FM	TK

