


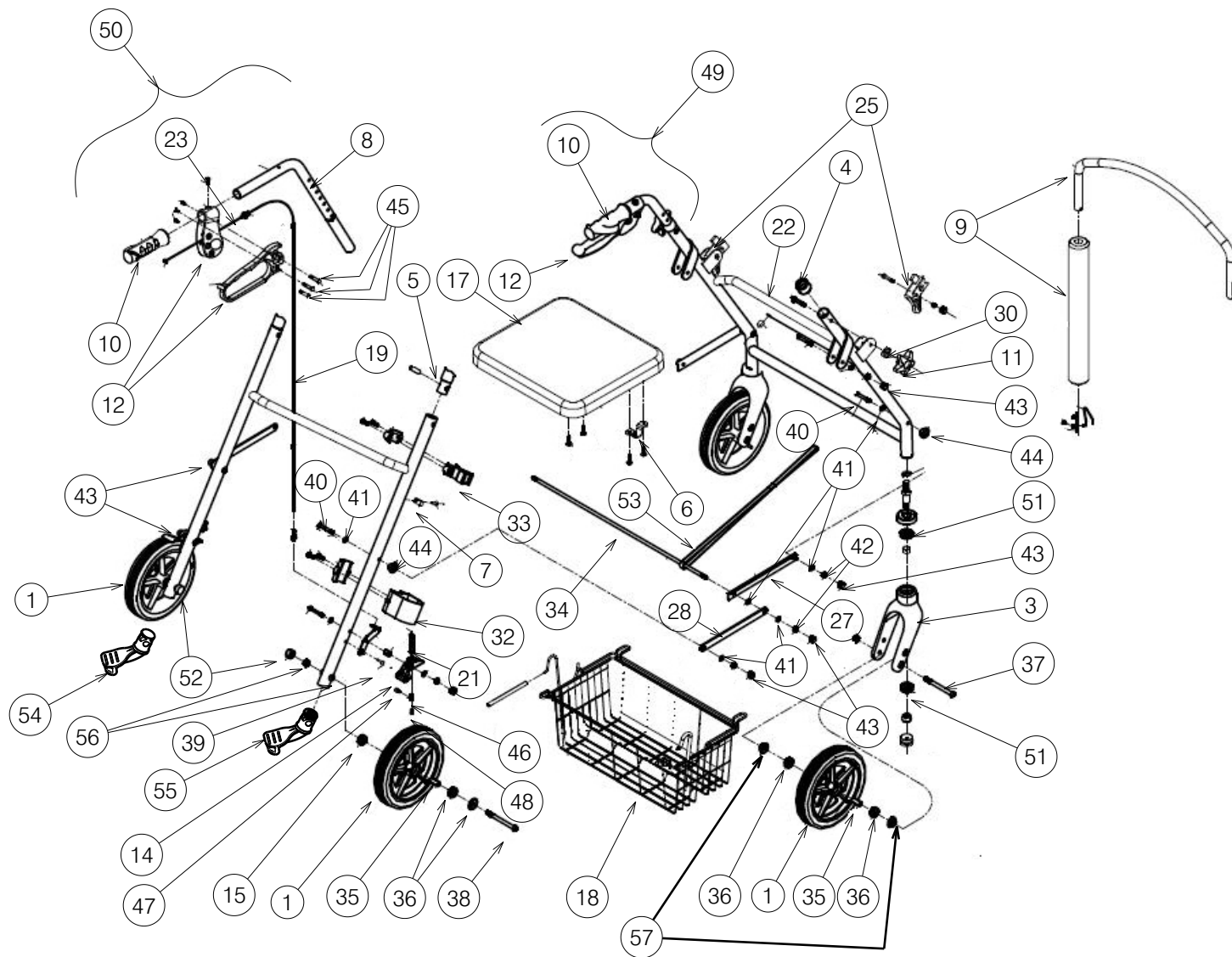

8002997

E000893 / O

ROLLATOR ECOLIGHT II T				Plan No.
Sheet 1/2				E000893 / O
Scale: 1:5	Drafted	Verified	Confirmed	
Date	30/10/2013	17/06/2025	17/06/2025	
Initials	SD	DVB	PJ	

Material							
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity
1	---	1911601	4	42	M6 DIN985	1001680	6
3	---	1909572	2	43	M6	1005554	12
4	---	1908896	2	44	Ø22	1909590	4
5	---	1908897	2	45	---	1915392	6
6	---	1909573	2	46	---	1916114	2
7	---	1909574	2	47	---	1916115	2
8	---	1908900	1	48	---	1916116	2
9	---	1909575	1	49	Left	1916074	1
10	---	1904435 (R)	1	50	Right	1916075	1
	---	1904436 (L)	1	51	---	1915599	4
11	---	1909576	2	52	M8	1003295	2
12	---	1909113	2	53	---	1912668	1
14	---	1909583	2	54	Left	1917890	1
15	Ø14x8.4x1	1909584	2	55	Right	1917891	1
17	---	1909585	1	56	---	1918592	4
18	---	1909586	1	57	---	1922652	2
19	---	1909587	2				
21	---	1909588	2				
22	M6x32	1908909	2				
23	---	1906542	2				
25	---	1909589	2				
27	---	1908913	2				
28	---	1908914	2				
30	---	1909590	2				
32	---	1909131	1				
33	---	1909591	1				
34	---	1908941	1				
35	---	1909592	4				
36	---	1909593	8				
37	---	1903978	2				
38	---	1903995	2				
39	---	1907154	2				
40	M6x40	1006111	4				
41	M6 DIN125	1001564	12				

8002997

ROLLATOR ECOLIGHT II T				Plan No.
Sheet 2/2				E000893/ O
Scale: 1:5	Drafted	Verified	Confirmed	
Date	30/10/2013	17/06/2025	17/06/2025	
Initials	SD	DVB	PJ	

E000893 / O