


Complete set 1914373

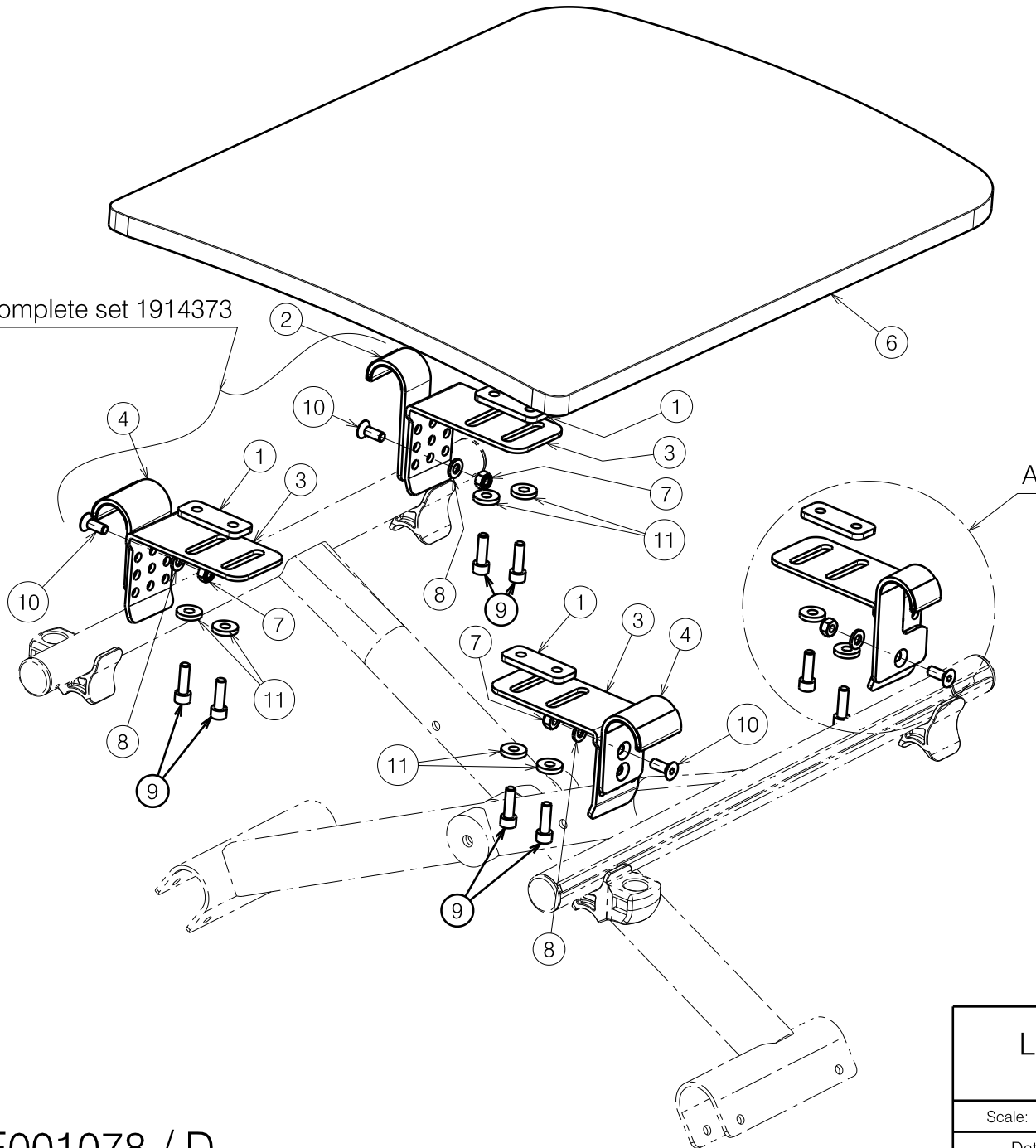

Material			
No.	Plan	Article	Quantity
1	Y021630	2013104	4
2	Rear Right	1914359	1
3	Y022626	2043504	4
4	Front	1914357	2
5	Rear Left	1914358	1
6	Size 39	1914364	1
	Size 42	1914365	1
	Size 44	1914366	1
	Size 46	1914367	1
	Size 48	1914368	1
	Size 50	1914369	1
	Size 53	1914370	1
	Size 56	1914371	1
	Size 60	1914372	1
7	M6 DIN985	1001680	4
8	M6 DIN125A	1001564	4
9	M6x20 DIN912	1001493	8
10	M6x16 DIN7991	1994752	4
11	M6 DIN7349	1003321	8

E001078 / D

L16/L45 ADJUSTABLE SEAT V-SERIES

Sheet 1/2

Plan No.

E001078 / D

Scale: 1:4

Drafted

Verified

Confirmed

Date

25/09/2017

17/06/2021

17/06/2021


Initials

SD


SD

PJ




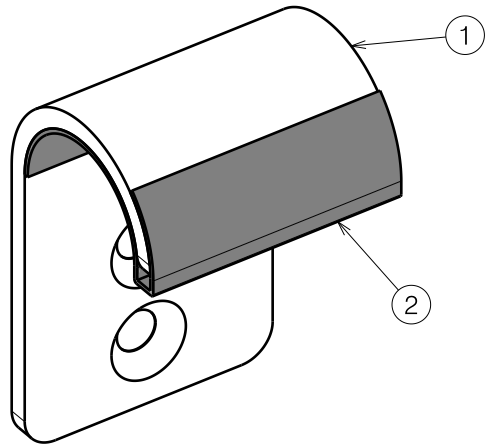

1914357 Front Hook

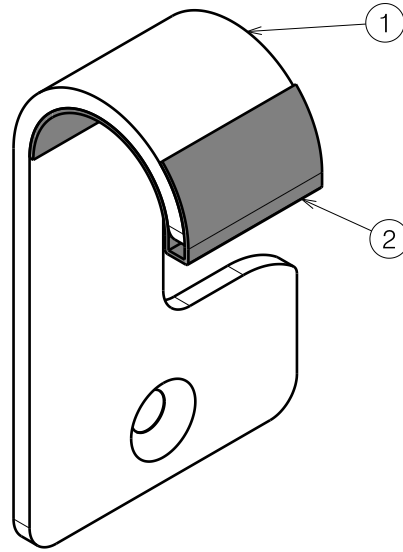

1914358 Rear Left Hook

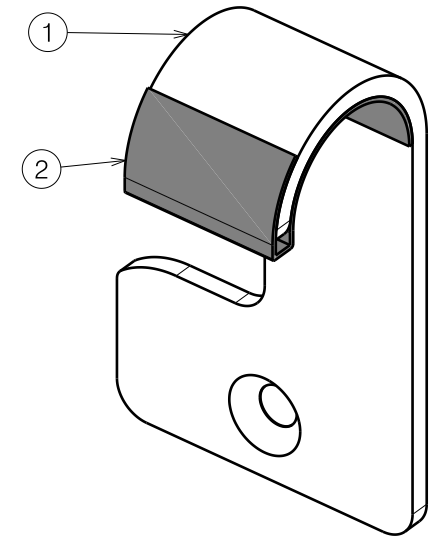


1915359 Rear Right Hook

1914357			
No.	Plan	Article	Quantity
1	Y022627	2043604	1
2	X005747	1914354	1

1914358			
No.	Plan	Article	Quantity
1	Y022628	2043704	1
2	X005748	1914355	1

1914359			
No.	Plan	Article	Quantity
1	Y022316	2043804	1
2	X005748	1914355	1

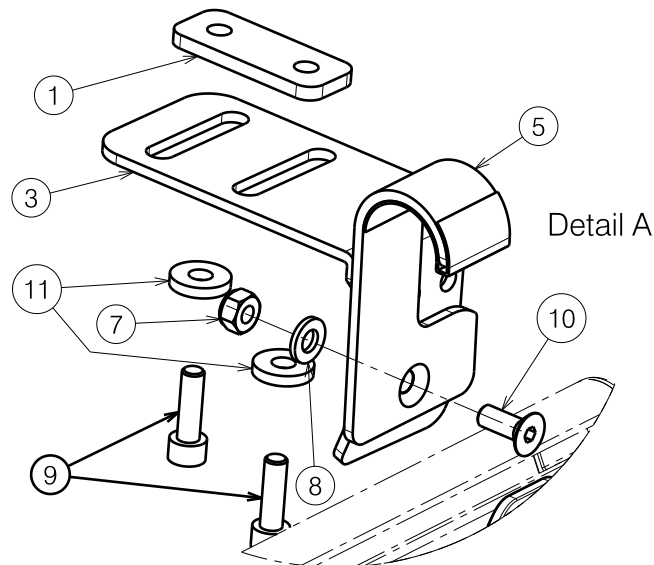

E001078 / D

L16/L45 ADJUSTABLE SEAT V-SERIES				Plan No.
Sheet 2/2				E001078 / D
Scale: 1:4	Drafted	Verified	Confirmed	
Date	25/09/2017	17/06/2021	17/06/2021	
Initials	SD	SD	PJ	