


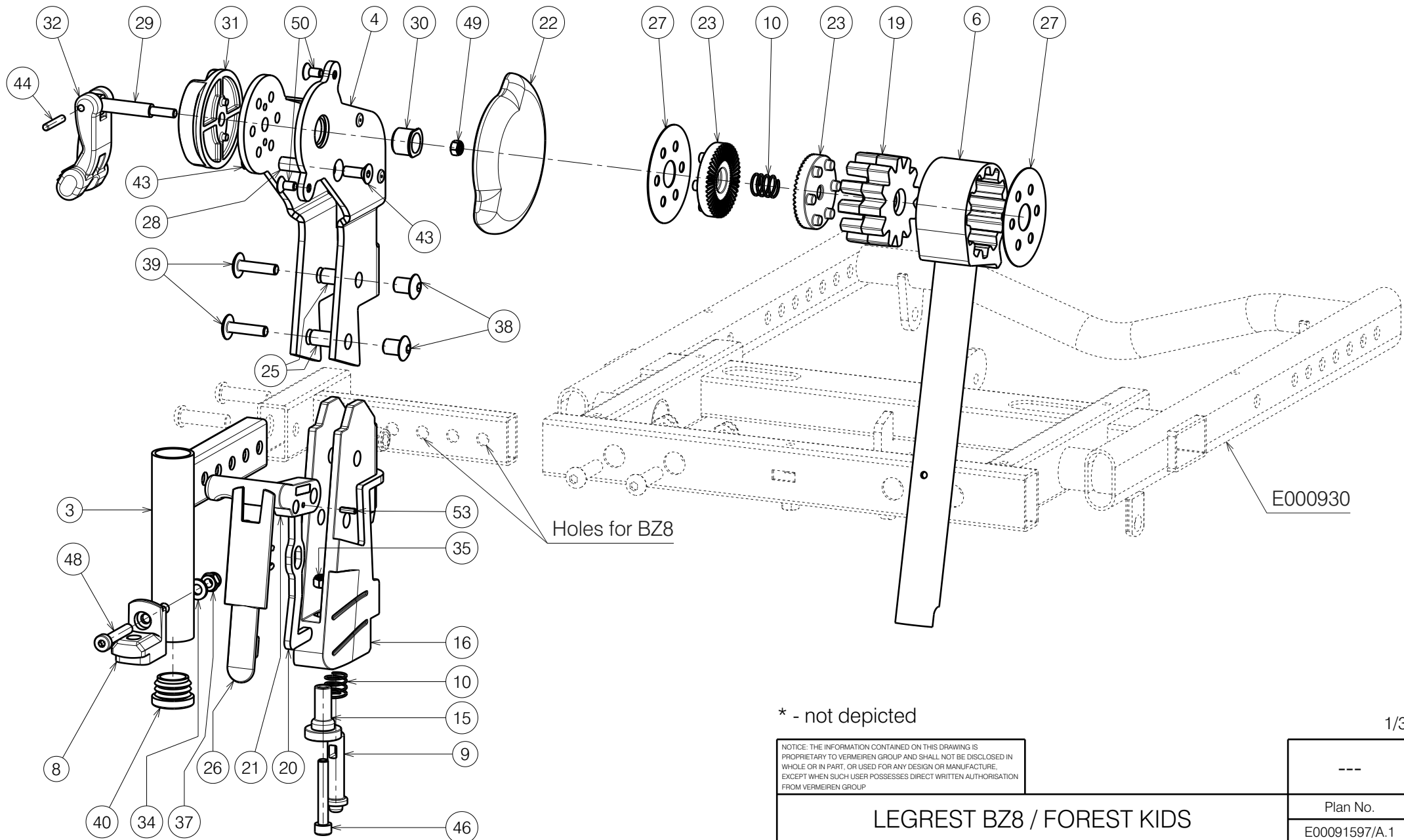


* - not depicted

1/3

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

LEGREST BZ8 / FOREST KIDS				Plan No.
				E00091597/A.1
Scale: 1:3	Drafted	Verified	Confirmed	
Date	25.11.2024	26/11/2024	26/11/2024	
Initials	AZ	AS	MK	

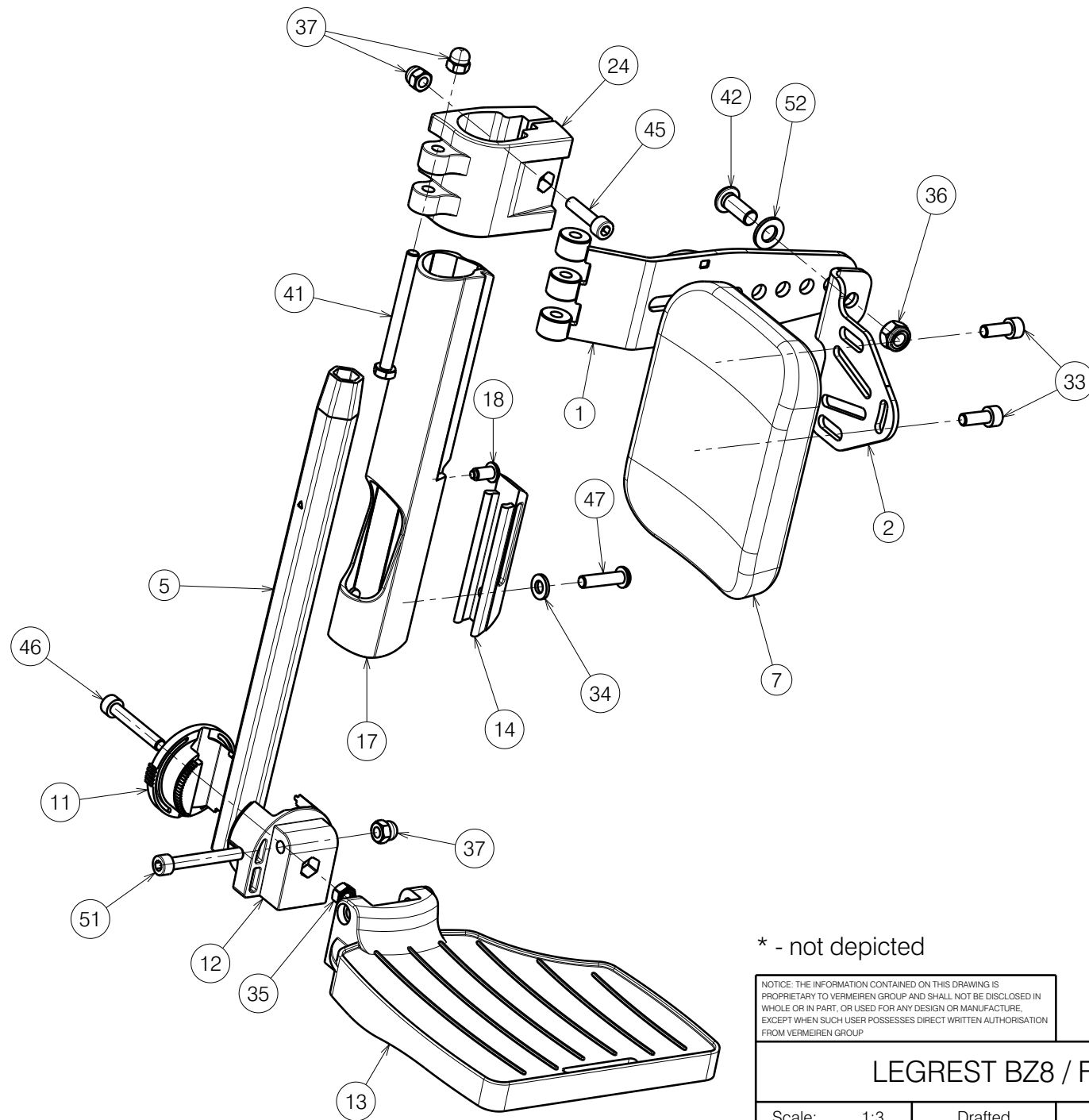

* - not depicted

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

LEGREST BZ8 / FOREST KIDS

Scale:	1:3	Drafted	Verified	Confirmed
Date	25.11.2024	26/11/2024	26/11/2024	
Initials	AZ	AS	MK	

2/3

Plan No.

E00091597/A.1



Only right side depicted

No.	Plan	Article	Qty	No.	Plan	Article	Qty	No.	Plan	Article	Qty
1	P00044872	2659754	1	29	X005028	G103900	1	45	M6x25 DIN912	1001494	1
2	P00089772(L)* P00089773(R)	2694354* 2694454	1* 1	30	X005046	G103800	1	46	M6x40 DIN912	1003909	2
3	P00091585	2694254	1	31	X005047	1908420	1	47	M6x25	1006998	1
4	Y018551(L)* Y018552(R)	1771654 1771554	1 1	32	X005055	1917169	1	48	M6x35 DIN7984	1903903	1
5	YP06404	2730199	1	33	M6x16 DIN912	1001492	2	49	M5 DIN985	1001679	1
6	YP09153	2636654	1	34	D6,4 DIN125A	1001564	2	50	M5x10 DIN7991	1999977	2
7	X003779	1900228	1	35	M6 DIN985	1001680	2	51	M6x45 DIN912	1003109	1
8	X004415	17193	1	36	M8 DIN985	1001682	1	52	X001119	1001554	1
9	X004417	1719200	1	37	M6x12 DIN986	1008492	4	53	Spring Pin 3x10	1008674	1
10	X004423	1717300	2	38	Cap Nut M6 RT	1905500	2				
11	X004464	1904212	1	39	M6x25 KT	1905501	2				
12	X004465	1904213	1	40	Cap D25	1991857	1				
13	X004570(L)* X004571(R)	1904646 1904647	1 1	41	M6x65 DIN931	1991860	1				
14	X004658	1904968	1	42	M8x20	1992349	1				
15	X004660	1905195	1	43	M6x16 DIN7991	1994752	2				
16	X004665	1905744	1	44	D2x12 DIN1475	1992597	1				
17	X004666	1905731	1								
18	X004687	1905639	1								
19	X004690	1905735	1								
20	X004692	1906013	1								
21	X004701(L)* X004710(R)	1905737 1905738	1 1								
22	X004713	1905734	1								
23	X004736	1905736	2								
24	X004739	1905739	1								
25	X004742	1739500	2								
26	X004773	1905885	1								
27	X004795	1905970	2								
28	X005025	1772000	1								

Only right side depicted

* - not depicted

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

LEGREST BZ8 / FOREST KIDS			
Scale: 1:5	Drafted	Verified	Confirmed
Date	25.11.2024	26/11/2024	26/11/2024
Initials	AZ	AS	MK