


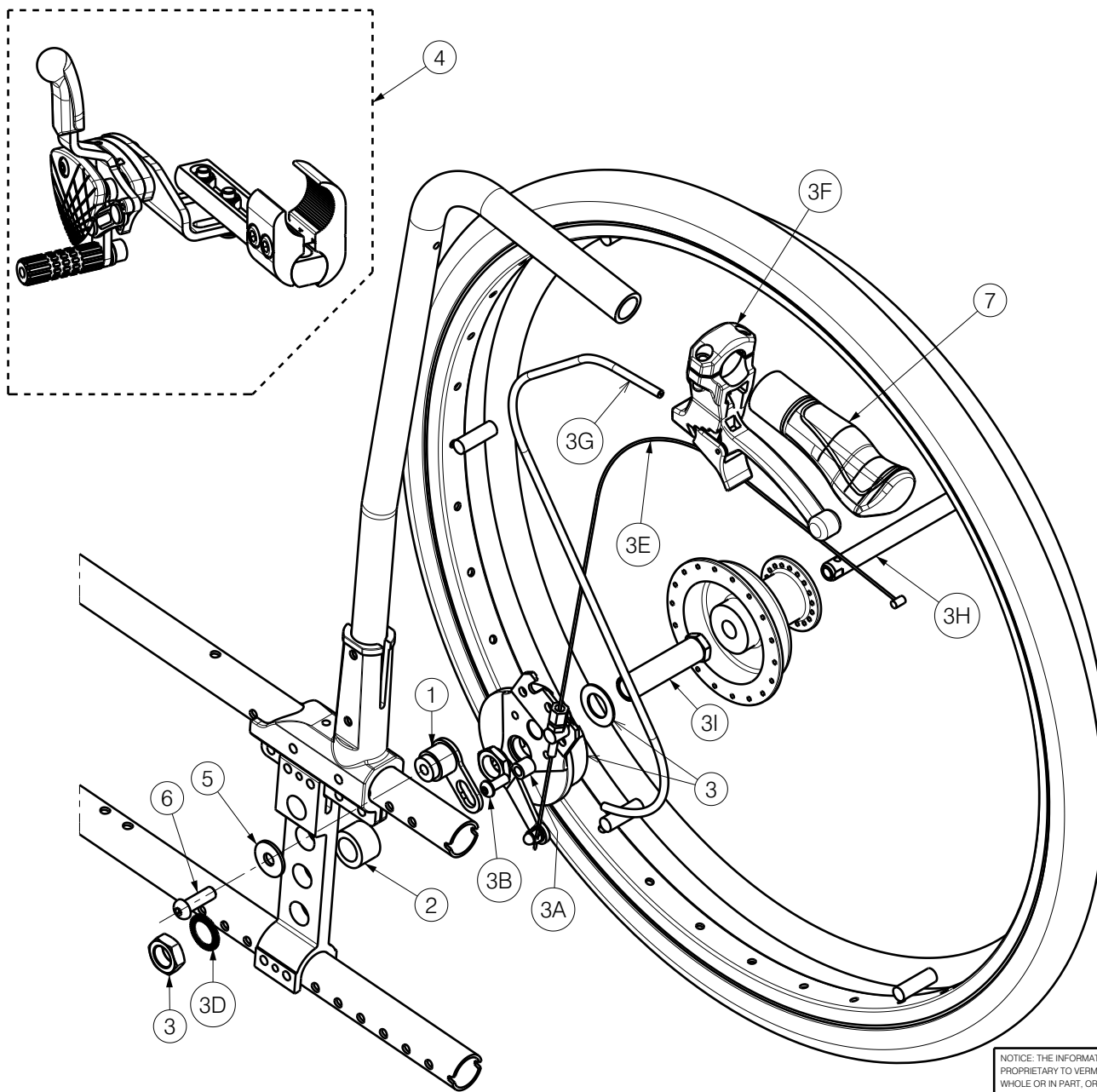

No.	Plan N°	Article N°	Quantity
1	Y016096	17378	2
2	X003986	1902199	2
3	D22	1906521 (L)	1
	D22	1906522 (R)	1
3A	X005545	2019900	2
3B	M6X16 (ISO7380)	1008459	2
3C*	--	1004109	2
3D	M16 type S (DIN17222)	1911653	2
3E	125cm	1004094	2
3F	D22	1994012	2
3G	--	1993218	(X2) 0.65mm
3H	D12 B74	1906205	2
3I	X005544	1912111	2
4	--	1921016 (L)	1
	--	1921017 (R)	1
5	D24XD8.4X2(DIN9021)	1996574	1
6	M8X25	1990501	1
7	X005513	1992600	2
* Not depicted			

* Not depicted
Only the right side is depicted
Quantities are for one pair

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

V500 ACTIVE B74				Plan No.
				E00103185/A1.1
Scale: 1:4	Drafted	Verified	Confirmed	
Date	14-04-2025	14-04-2025	14-04-2025	
Initials	AR	MP	TK	